SpinFOB

Colorectal Cancer (CRC) is a worldwide problem, with an annual incidence of approximately 1 million cases and an annual mortality of more than 500,000. The absolute number of cases will increase over the next two decades as a result of the aging and expansion of populations in both the developed and developing countries. CRC is the second most common cause of cancer mortality among men and women. Colorectal cancers may shed blood and other tissue components that can be detected in the feces long before the development of clinical symptoms¹.

The 2010 European guidelines for quality assurance in colorectal cancer screening and diagnosis recommended the adoption of quantitative FIT (Fecal Immunochemical Testing) as the primary screening modality for colorectal cancer^{2,3}. The merits of FIT have been also promoted by the World Endoscopy Organization Colorectal Cancer Screening Committee's Expert Working Group on 'FIT for Screening'.

The new Spinreact FOB kit for the detection of Fecal Occult Blood is based on polyclonal antibodies, allowing quantitative screening of CRC without diet restrictions on a dedicated automated analyzer. Sample collection may be performed at home or at the lab thanks to the collection tube with hemoglobin stabilizers.

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References / Referencias:

1. World Gastroenterology Organisation/International Digestive Cancer Alliance Practice Guidelines: Colorectal cancer screening. Ed. 2007.
2. European guidelines for quality assurance in colorectal cancer screening and diagnosis, First edition ed., Publications Office of the European Union, Luxembourg, 2010.
3. Halloran, S.P., Launoy, G., Zappa, M. European guidelines for quality assurance in colorectal cancer screening and diagnosis. First Edition. Faecal occult blood testing, *Endoscopy* 44 (2012) SE65-SE87.

Sample Collection

Optimized sample collection system

Standardized Feaces volume

Sample is obtained using the same primary tube that will be used for the analysis of the different available parameters

Sample stability up to 7 days

Spinreact Turbidimetric Assay

40 years experience of manufacturing turbidimetric reagents

Without prozone effect up to Hb 100000ng/mL at the 100ng/mL cutoff

No interference with bilirubin, albumin or exogenous animal hemoglobin

No need of dietary restrictions

Dedicated System

Refrigerated samples and reagents on an optimized rotor that allows to place three turbidimetric techniques, washing solutions and 28 samples

Other techniques like H pilory and Calprotectin are being developed to be applied on feaces using the same platform

Calibrator and Controls

Freeze-dried

18 months of shelf life

Stability after reconstitution 24h at 2°C - 8°C or 1 month at -20°C

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