Detergent for Spincell 5 Series IVD

INTENDED USE

Detergent is used with matching analyzer for daily maintenance and flow system and probe cleaning.

FUNCTIONS

With the functions of emulsification and catalysis, the Detergent emulsifies the fat, distributes the solidified dirt and transforms the protein into water-soluble amino-acids to maintain the cleanness of tubes and flow-cells. With effective humidification, it also discharges the bubbles from the entire flow system and ensures a normal working performance. The Detergent will not stain, corrupt the tubes or shorten instrument's service life.

INGREDIENTS

Preservative Solution	2 g/L
Chloride Sulfate	20 g/L
Buffer Solution	1 g/L
Anticoagulant	1~2 g/L
Surfactant	3 g/L

STORAGE & STABILITY

This product will be stable for 24 months when stored unopened in a dark place to avoid sunlight. Once opened, this product should be consumed within 60 days and stored below 35°C.

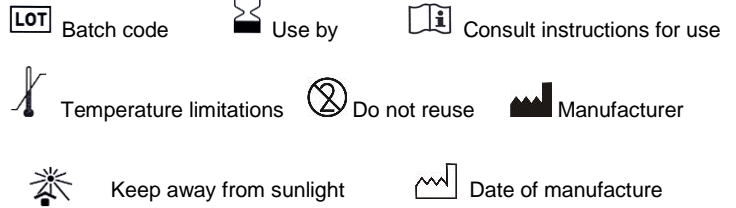
OPERATIONS

1. Place the Detergent container beneath the analyzer to avoid direct sunlight. Keep away from cooling or heating air outlet.
2. Use it at 15°C~30°C.
3. Connect Detergent container with the analyzer according to Operation Manual.

RESTRICTIONS, PRECAUTIONS & NOTICE

1. Use the reagent under the professional technician's guidance.
2. Make sure that the reagent matches with your analyzer. Any mismatch will lead to misleading results.
3. Avoid contacting with skin and eyes. In case of contact, rinse immediately with plenty of water and seek medical advice.
4. Avoid inhaling the evaporated reagent.
5. Tight up the container's cap to prevent from volatilization and contamination.
6. Once opened, dispose the remains over 60 days.
7. If product has been frozen, let product warm to room temperature, and then gently invert product to mix it completely. Verify background results before use.

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