

COVID-19 Testing

Antigen rapid test

Nasopharyngeal, Oropharyngeal
and Nasal samples
Results in 15 minutes
Kit includes all components needed
Individual buffer vials

IgG / IgM rapid test

Total blood, serum and
plasma sample
Results in 10 minutes
Single use pipette included



In collaboration with **KEWEI** Diagnostics


IgG / IgM test



Procedure

- Sample:** Fingerpick / Total blood / Serum / Plasma
- Testing Time:** 10 minutes
- Volume:**
- Total Blood: 20µl / 1 drop
 - Plasma: 10µl
 - Serum: 10µl

Interpretation:

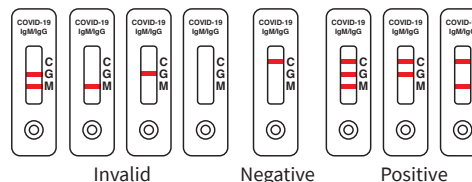

Accuracy data*:

	IgM	IgG
Sensibility	95,7 %	91,8 %
Specificity	97,3 %	96,4 %
Global accuracy	93,8 %	95,0 %


*Data included on manufacturer's specifications. Please refer to the IFU for further information.

Antigen test




Procedure

- Sample:** Nasopharyngeal swab (included), Oropharyngeal and Nassal swab
- Testing Time:** 15 minutes
- Sampling:**

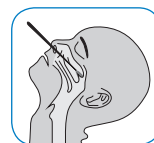


Nasopharyngeal

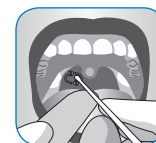


Oropharyngeal

Testing procedure:

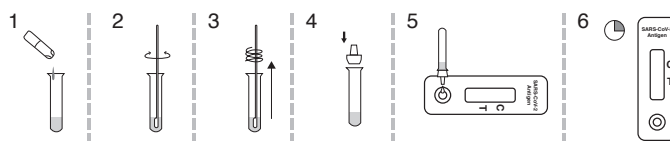

- 1- Place the buffer in the plastic tube
- 2- Twist the swab x10 in contact with the buffer
- 3- Discard the swab
- 4- Place the dropper cap on the plastic tube
- 5- Apply 2 drops / 60ul to the sample well
- 6- Wait 15 minutes and interpret the result

Interpretation:

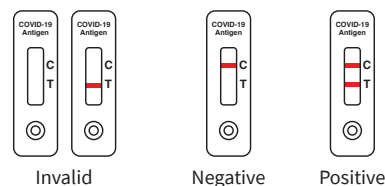


Accuracy data*:


	Antigen
Sensibility	95,52 %
Specificity	100 %
Global accuracy	97,64 %

*Data included on manufacturer's specifications. Please refer to the IFU for further information.

Evolution of the different parameters during the progression of the infection

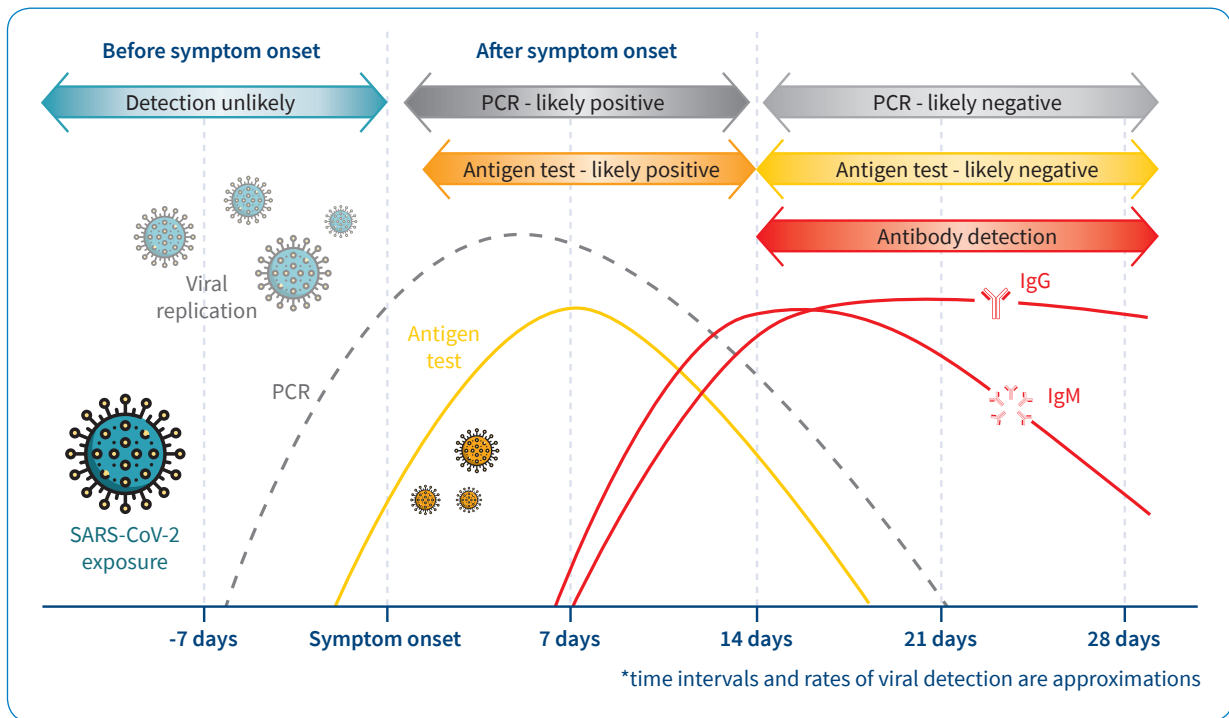

Important parameters in COVID patients monitoring. SARS-CoV-2 Effect and relevance

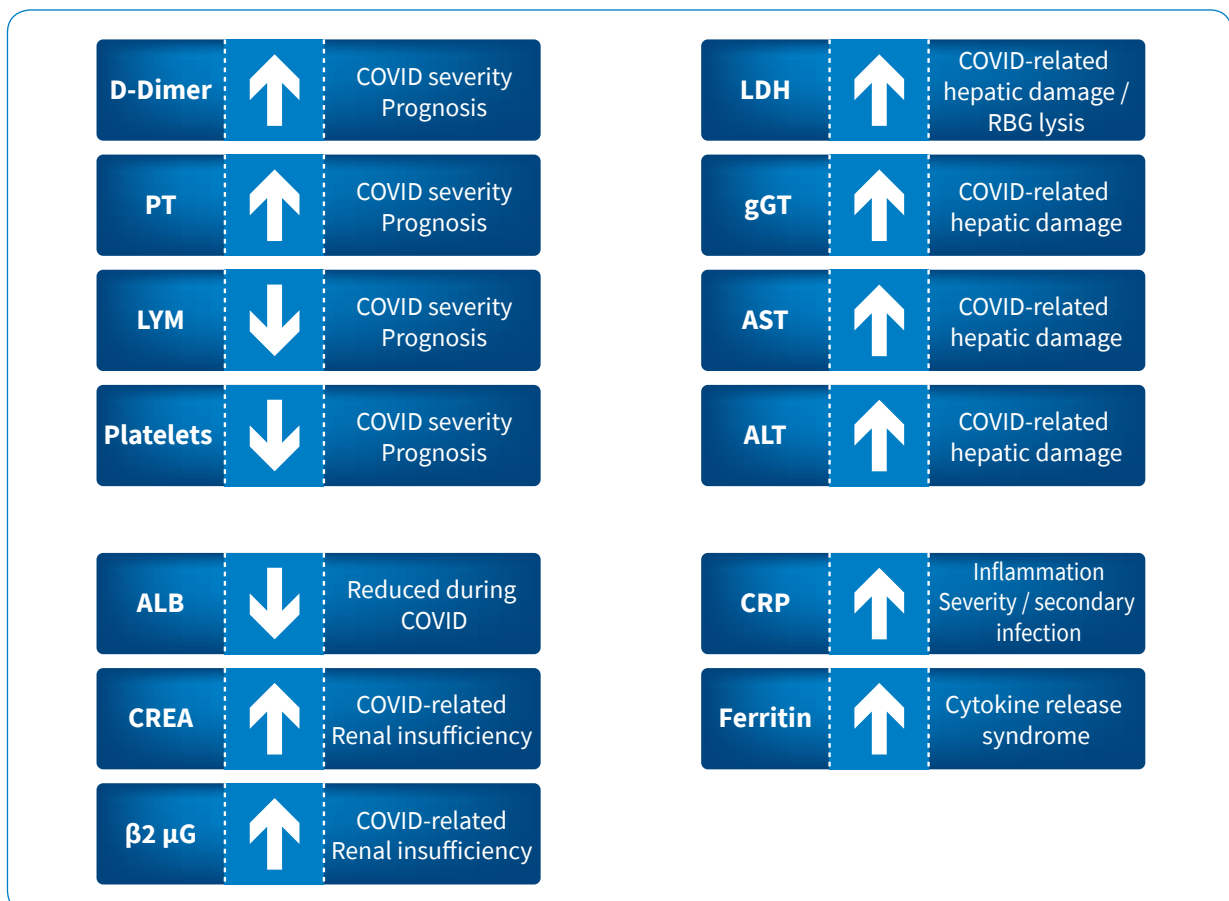

Antigen and IgG + IgM COVID rapid test

- **Fast:** Avoid waiting for days, get the results in 10 to 15 minutes.
- **Simple:** Everything needed for the determination is included in the kit*.
- **Convenient:** No special equipment, storage, or transportation conditions required.
- **Accurate:** A good screening method prior to more costly alternatives.
- **Facilitates testing:** Low cost and convenience help to increase the number of people being tested.
- **Individual buffer vials for antigen test.**

* In case of analyzing fingerprick total blood not included lancets will be required.

Code	Description	Test per kit
1504078	COVID IgG/IgM kit	25
1505079	COVID Antigen kit	25
1504075	Spin-Influenza A+B	20
1504080	Spin - RSV	20
1504090	Resp. Adenovirus	20

Distributor · Distribuidor

Spinreact SAU
Ctra Sta. Coloma, 7
17176 St. Esteve d'en Bas
Girona, Spain
Tel. +34 972 690 800
Fax +34 972 690 099
spinreact@spinreact.com

Commercial Office
C/ Plató, 6
08021 Barcelona, Spain
Tel: +34 93 200 35 44
Fax: +34 93 209 72 62
comercial@spinreact.com
www.spinreact.com

