**Multidrug / Multidroga**  
**High Calibrator Urine / Calibrador Orina Alto**  
 DoA Liquid Test

**Multidrug Calibrator High Cut-off for Urine IVD**

Store at 2 - 8°C.

**INTENDED USE**

The Multidrug calibrator is used for the calibration of the following Spinreact DoA-EIA qualitative liquid tests at high cutoff levels: Amphetamine, Barbiturates, Cocaine metabolite, Methadone, Opiates.

**REAGENTS**

Ref. 932915	Multidrug Calibrator High Cut-off <b>Human Urine. Sodium azide 0.5 g/L</b> <b>Methamphetamine, Secobarbital, Benzoyllecgonine, Methadone, Morphine.</b>
-------------	---

The concentration of each drug in this calibrator is:

	ng/mL
Methamphetamine	1000
Secobarbital	200
Benzoyllecgonine	300
Methadone	300
Morphine	2000

**PREPARATION AND STABILITY**

The calibrator is ready to use. No calibrator preparation is required. The calibrator is stable until the expiration date on the label when stored tightly closed at 2-8°C and contaminations are prevented during their use.

**WARNING AND PRECAUTIONS**

- The calibrator is for in vitro diagnostic use only. Harmful if swallowed.
- The calibrator contains sodium azide that may react with lead or copper plumbing to form potentially explosive metal azide. When disposing such calibrator or wastes always flush with a large volume of water to prevent azide build-up.
- Do not use the calibrator beyond their expiration date.
- Keep all containers closed when not in use to avoid microbial contamination.
- Do not mix calibrators from different manufacturers.
- Do not freeze the calibrator.

Handle all urine specimens as if they were potentially infectious.

**PROCEDURE**

The Multidrug calibrator should be used in the following Urine reagents: Amphetamine, Barbiturates, Cocaine metabolite, Methadone, Opiates.

**REFERENCES**

1. Urine testing for Drug of Abuse. National Institute on Drug Abuse (NIDA) Research Monograph 73, 1986.
2. European Laboratory Guidelines for Legally Defensible Workplace Drug Testing – Urine Drug Testing, EWDTS, 2002.
3. Spinreact DoA EIA package inserts.

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**REFERENCIAS**

1. Urine testing for Drug of Abuse. National Institute on Drug Abuse (NIDA) Research Monograph 73, 1986.
2. European Laboratory Guidelines for Legally Defensible Workplace Drug Testing – Urine Drug Testing, EWDTS, 2002.
3. Hoja de instrucciones de los kits Spinreact DoA-EIA.

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Ref. 932915 **CONT.** 1x10 mL