Multidrug / *Multidroga* High Cut-Off Control Urine - *Control Cut-off Alto Orina* DoA Liquid Test

Multidrug High Control Cut-off for Urine IVD

Store at 2 - 8°C.

INTENDED USE

The Multi-Drug Control is consisting of two levels $\pm 25\%$ from the high cutoffs for the control of the following Spinreact DoA-EIA liquid tests: Amphetamine, Barbiturates, Cocaine metabolite, Methadone, Opiates.

REAGENTS

Level - 25 %	Multidrug High Control Cut-off Human Urine. Sodium azide 0.5 g/L. Methamphetamine, Secobarbital, Benzoylecgonine, Methadone, Morphine.
Level + 25 %	Multidrug High Control Cut-off Human Urine. Sodium azide 0.5 g/L. Methamphetamine, Secobarbital, Benzoylecgonine, Methadone, Morphine.

Each level has the following concentrations:

ng/ml	- 25%	+ 25%
Methamphetamine	750	1250
Secobarbital	150	250
Benzoylecgonine	225	375
Methadone	225	375
Morphine	1500	2500

PREPARATION AND STABILITY

The controls are ready to use. No control preparation is required. The control are stable until the expiration date on the label when stored tightly closed at 2-8°C and contaminations are prevented during their use.

WARNING AND PRECAUTIONS

- The controls are for in vitro diagnostic use only. Harmful if swallowed.
 - The controls contain sodium azide that may react with lead or copper plumbing to form potentially explosive metal azide. When disposing such controls or wastes always flush with a large volume of water to prevent azide build-up.
 - Do not use the controls beyond their expiration dates.
 - Keep all containers closed when not in use to avoid microbial contamination.
 - Do not mix controls from different manufacturers.
 - Do not freeze controls.
- Handle all urine specimens as if they were potentially infectious.

PROCEDURE

The controls should be used in the following Urine reagents: Amphetamine, Barbiturates, Cocaine metabolite, Methadone, Opiates.

REFERENCES

1. Urine testing for Drug of Abuse. National Institute on Drug Abuse (NIDA) Research Monograph 73, 1986.
2. European Laboratory Guidelines for Legally Defensible Workplace Drug Testing – Urine Drug Testing, EWDTS, 2002.
3. Spinreact DoA- EIA package inserts.

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REFERENCIAS

1. Urine testing for Drug of Abuse. National Institute on Drug Abuse (NIDA) Research Monograph 73, 1986.
2. European Laboratory Guidelines for Legally Defensible Workplace Drug Testing – Urine Drug Testing, EWDTS, 2002.
3. Hojas de instrucciones de los kits Spinreact Do-EIA.

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Ref. 935915 CONT. Nivel -25 %: 2x10 mL
Nivel +25%: 2x10 mL