Multidrug / *Multidroga* Low Cut-Off Control Urine - *Control Cut-off Bajo Orina* DoA Liquid Test

Multidrug Low Control Cut-off for Urine IVD

Store at 2 - 8°C.

INTENDED USE

The Multi-Drug Control is consisting of two levels $\pm 25\%$ from the low cutoffs for the control of the following Spinreact DoA-EIA liquid tests: Amphetamine, Barbiturates, Cocaine metabolite, Methadone, Opiates.

REAGENTS

Level - 25 %	Multidrug Low Control Cut-off Human Urine. Sodium azide 0.5 g/L. Methamphetamine, Secobarbital, Benzoylecgonine, Methadone, Morphine.
Level + 25 %	Multidrug Low Control Cut-off Human Urine. Sodium azide 0.5 g/L. Methamphetamine, Secobarbital, Benzoylecgonine, Methadone, Morphine.

Each level has the following concentrations:

ng/ml	- 25%	+ 25%
Methamphetamine	225	375
Secobarbital	150	250
Benzoylecgonine	112,5	187,5
Methadone	112,5	187,5
Morphine	225	375

PREPARATION AND STABILITY

The controls are ready to use. No control preparation is required. The controls are stable until the expiration date on the label when stored tightly closed at 2-8°C and contaminations are prevented during their use.

WARNING AND PRECAUTIONS

- The controls are for in vitro diagnostic use only. Harmful if swallowed.
 - The controls contain sodium azide that may react with lead or copper plumbing to form potentially explosive metal azide. When disposing such controls or wastes always flush with a large volume of water to prevent azide build-up.
 - Do not use the controls beyond their expiration dates.
 - Keep all containers closed when not in use to avoid microbial contamination.
 - Do not mix controls from different manufacturers.
 - Do not freeze controls.
- Handle all urine specimens as if they were potentially infectious.

PROCEDURE

The controls should be used in the following Urine reagents: Amphetamine, Barbiturates, Cocaine metabolite, Methadone, Opiates.

REFERENCES

1. Urine testing for Drug of Abuse. National Institute on Drug Abuse (NIDA) Research Monograph 73, 1986.
2. European Laboratory Guidelines for Legally Defensible Workplace Drug Testing – Urine Drug Testing, EWDTS, 2002.
3. Spinreact DoA- EIA package inserts.

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REFERENCIAS

1. Urine testing for Drug of Abuse. National Institute on Drug Abuse (NIDA) Research Monograph 73, 1986.
2. European Laboratory Guidelines for Legally Defensible Workplace Drug Testing – Urine Drug Testing, EWDTS, 2002.
3. Hojas de instrucciones de los kits Spinreact DoA-EIA.

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Ref. 935910 **CONT.** Nivel -25 %: 2x10 mL
Nivel +25%: 2x10 mL