Ecstasy / Éxtasis Calibrador 500 Urine / Calibrador 500 Orina DoA Liquid Test

Ecstasy Calibrator (Cut-off 500 ng/mL) for Urine IVD

Store at 2 - 8°C.

INTENDED USE

The Ecstasy Calibrator is used for the calibration of Spinreact DoA-EIA qualitative liquid test for Ecstasy at the cutoff level of 500ng/mL.

REAGENTS

Ref. 932935	Ecstasy Calibrator cut-off 500 ng/mL. Human Urine. Sodium azide 0.5 g/l. MDMA
-------------	--

The concentration of drug (3,4-methylenedioxymethamphetamine) in this calibrator is:

	ng/mL
MDMA	500

PREPARATION AND STABILITY

The calibrator is ready to use. No calibrator preparation is required. The calibrator is stable until the expiration date on the label when stored tightly closed at 2-8°C and contaminations are prevented during their use.

WARNING AND PRECAUTIONS

- This calibrator is for in vitro diagnostic use only. Harmful if swallowed.
- The calibrator contains sodium azide that may react with lead or copper plumbing to form potentially explosive metal azide. When disposing such calibrators or wastes always flush with a large volume of water to prevent azide build-up.
- Do not use the calibrator beyond their expiration dates.
- Keep all containers closed when not in use to avoid microbial contamination.
- Do not mix calibrators from different manufacturers.
- Do not freeze the calibrator.

Handle all urine specimens as if they were potentially infectious.

PROCEDURE

The calibrator should be used in the Ecstasy reagent for urine.

REFERENCES

1. Urine testing for Drug of Abuse. National Institute on Drug Abuse (NIDA) Research Monograph 73, 1986.
2. European Laboratory Guidelines for Legally Defensible Workplace Drug Testing – Urine Drug Testing, EWDTS, 2002.
3. Spinreact MDMA- EIA (Refs #930058/930060) package inserts.

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REFERENCIAS

1. Urine testing for Drug of Abuse. National Institute on Drug Abuse (NIDA) Research Monograph 73, 1986.
2. European Laboratory Guidelines for Legally Defensible Workplace Drug Testing – Urine Drug Testing, EWDTS, 2002.
3. Hoja de instrucciones del kit Spinreact MDMA-EIA (Refs # 930058/930060).

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Ref. 932935 **CONT.** 1x10 mL