Benzodiazepines / Benzodiazepinas

High Calibrator 300 Urine / Calibrador Orina Alto 300

DoA Liquid Test

Benzodiazepine Calibrator High Cut-off (300 ng/mL) for Urine IVD

Store at 2 - 8°C.

INTENDED USE

The Benzodiazepine Calibrator is used for the calibration of Spinreact DoA-EIA qualitative liquid test for Benzodiazepines at the high cutoff level (300ng/mL).

REAGENTS

Ref. 933082	Benzodiazepine Calibrator High Cut-off (300 ng/mL) Human Urine. Sodium azide 0.5 g/L Oxacepam
-------------	---

The concentration of drug in this calibrator is:

	ng/mL
Oxacepam	300

PREPARATION AND STABILITY

The calibrator is ready to use. No calibrator preparation is required. The calibrator is stable until the expiration date on the label when stored tightly closed at 2-8°C and contaminations are prevented during their use.

WARNING AND PRECAUTIONS

- The calibrator is for in vitro diagnostic use only. Harmful if swallowed.
- The calibrator contains sodium azide that may react with lead or copper plumbing to form potentially explosive metal azide. When disposing such calibrator or wastes always flush with a large volume of water to prevent azide build-up.
- Do not use the calibrator beyond their expiration date.
- Keep all containers closed when not in use to avoid microbial contamination.
- Do not mix calibrators from different manufacturers.
- Do not freeze the calibrator.

Handle all urine specimens as if they were potentially infectious.

PROCEDURE

The calibrator should be used in the Benzodiazepine reagent for urine.

REFERENCES

1. Urine testing for Drug of Abuse. National Institute on Drug Abuse (NIDA) Research Monograph 73, 1986.
2. European Laboratory Guidelines for Legally Defensible Workplace Drug Testing – Urine Drug Testing, EWDTS, 2002.
3. Spinreact Benzodiazepines EIA (Refs #930018/930020) package inserts.

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REFERENCIAS

1. Urine testing for Drug of Abuse. National Institute on Drug Abuse (NIDA) Research Monograph 73, 1986.
2. European Laboratory Guidelines for Legally Defensible Workplace Drug Testing – Urine Drug Testing, EWDTS, 2002.
3. Hoja de instrucciones del kit Spinreact Benzodiazepinas-EIA (Refs # 930088/930020).

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Ref. 933082 CONT. 1x10 mL