
Quantitative determination of lactate dehydrogenase (LDH) IVD

Store at 2-8°C

PRINCIPLE OF THE METHOD

Lactate dehydrogenase (LDH) catalyses the reduction of pyruvate by NADH, according to the following reaction: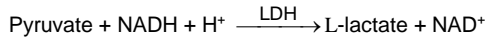

The rate of decrease in concentration of NADPH, measured photometrically, is proportional to the catalytic concentration of LDH present in the sample¹.

CLINICAL SIGNIFICANCE

Lactate dehydrogenase (LDH) is an enzyme with wide tissue distribution in the body.

The higher concentrations of LDH are found in liver, heart, kidney, skeletal muscle and erythrocytes.

Increased levels of the enzyme are found in serum in liver disease, myocardial infarction, renal disease, muscular dystrophy and anemia^{1,4,5}.

Clinical diagnosis should not be made on a single test result; it should integrate clinical and other laboratory data.

REAGENTS

Reagent 1	Imidazol	65 mmol/L
Buffer	Pyruvate	0,6 mmol/L
Reagent 2	NADH	0,18 mmol/L
Substrate		

PREPARATION

All the reagents are ready to use.

STORAGE AND STABILITY

All the components of the kit are stable until the expiration date on the label when stored tightly closed at 2-8°C, protected from light and contaminations prevented during their use.

Do not use reagents over the expiration date.

Signs of reagent deterioration:

- Presence of particles and turbidity.
- Blank absorbance (A) at 340 nm <1,00.

ADDITIONAL EQUIPMENT

- SPIN640 / SPIN640Plus Autoanalyzer.
- General laboratory equipment.

SAMPLES

Serum¹. Separated from cells as rapidly as possible. Do not use oxalates as anticoagulants since they inhibit the enzyme. Do not use haemolysed samples. Stability: 2 days at 2-8°C.

REFERENCE VALUES¹

230-460 U/L

These values are for orientation purpose; each laboratory should establish its own reference range.

QUALITY CONTROL

Control sera are recommended to monitor the performance of assay procedures: SPINROL H Normal and Pathologic (Ref. 1002120 and 1002210).

If control values are found outside the defined range, check the instrument, reagents and technique for problems.

Each laboratory should establish its own Quality Control scheme and corrective actions if controls do not meet the acceptable tolerances.

BARCODED REAGENTS LOAD MUST BE PRECEDED OF A SPINREACT "DATABASE" COPY INTO THE ANALYZER SOFTWARE. IT IS AVAILABLE UNDER REQUEST TO SPINREACT.

SPIN640 APPLICATION

TEST INFORMATION		REAGENT VOLUME	
Nº	**	Vol. R1	240
Test	LDH	Vol. R2	60
Full Name	LDH	Vol. R3	
Standard nº		Vol. R4	
SAMPLE VOLUME		RESULT SETUP	
Vol. Sample Stand.	5	Decimal	1 Slope 1
Vol. Sample Increas.		Unit	U/L Inter. 0
Vol. Sample Dec			
REACTION PARAMETERS			
Reac. Type	Kinetic	Direction	Decrease
Pri. Wave.	340	Reagent Blank	0-0
Sec. Wave.		React. Time	50-70

SPIN640Plus APPLICATION

EDIT PARAMETERS			
Test	LDH	No.	**
Full name	LDH	Print name	LDH
Reac. Type	Kinetic	Direction	Decrease
Pri. Wave.	340	Sec. Wave.	
Unit	U/L	Decimal	0.1
Reagent Blank	0 - 0	React. Time	57 - 77
Vol. Sample	5 ul	R1	240 ul
Increased		R2	60 ul
Decreased		R3	
Sample blank		R4	

The Calibration is stable until **32 days**. After this period the Calibration must be performed again in order to obtain good results.

PERFORMANCE CHARACTERISTICS

Measuring range: From detection limit of 3,42 U/L to linearity limit of 1600 U/L.

If the results obtained were greater than linearity limit, dilute the sample 1/10 with NaCl 9 g/L and multiply the result by 10.

Precision:

	Intra-assay (n=20)		Inter-assay (n=20)	
	Mean (U/L)	400	785	392
SD	3,15	10,97	6,23	9,93
CV (%)	0,79	1,40	1,59	1,28

Sensitivity: 1 U/L = 0,00009 ΔA/min.

Accuracy: Results obtained using SPINREACT reagents (y) did not show systematic differences when compared with other commercial reagents (x).

The results obtained using 50 samples were the following:

Correlation coefficient (r)²: 0,98382.

Regression equation: y= 0,8988x + 2,583.

The results of the performance characteristics depend on the analyzer used.

BIBLIOGRAPHY

1. Pesce A. Lactate dehydrogenase. Kaplan A et al. Clin Chem The C.V. Mosby Co. St Louis. Toronto. Princeton 1984; 1124-117, 438.
2. Young DS. Effects of drugs on Clinical Lab. Tests, 4th ed AACC Press, 1995.
3. Young DS. Effects of disease on Clinical Lab. Tests, 4th ed AACC 2001.
4. Burtis A et al. Tietz Textbook of Clinical Chemistry, 3rd ed AACC 1999.
5. Tietz N W et al. Clinical Guide to Laboratory Tests, 3rd ed AACC 1995.

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