α₁-Ac GLY

α₁-Acid Glycoprotein

Turbidimetry

Quantitative determination of α₁-Acid Glycoprotein (α₁-Ac GLY)

IVD

Store at 2-8°C

PRINCIPLE OF THE METHOD

Anti- α₁-Ac Gly antibodies when mixed with samples containing α₁-Ac Gly, form insoluble complexes. These complexes cause an absorbance change, dependent upon the α₁-Ac Gly concentration of the patient sample, that can be quantified by comparison from a calibrator of known α₁-Ac Gly concentration.

CLINICAL SIGNIFICANCE

α₁-Ac Glycoprotein (also known as orosomucoid) is a glycoprotein synthesized by hepatic parenchymal cells, but granulocytes and monocytes may also contribute significantly to plasma levels in sepsis. It has long been known to bind a large number of basic and lipophilic compounds (progesterone and related hormones).

It is an acute phase response protein that shows a 3 to 4-fold increase in most conditions associated with inflammation or tissue necrosis, and may be one of the most reliable indicators of clinical activity of ulcerative colitis. Levels also are increased by glucocorticoid effect.

Synthesis and plasma levels are decreased by estrogens.

REAGENTS

R 1 Diluent	Tris buffer 20 mmol/L, PEG 8000, pH, 8,3. Preservative.
R 2 Antibody	Goat serum, anti-human α ₁ -Ac Glycoprotein, pH 7,5. Preservative.
Optional	Ref: 1102003 PROT CAL.

CALIBRATION

The assay has been standardized against the Reference Material ERM-DA470k/IFCC. It must be used the PROT CAL to calibrate the reagent.

PREPARATION

Reagents: Ready to use.

STORAGE AND STABILITY

All the components of the kit are stable until the expiration date on the label when stored tightly closed at 2-8°C and contaminations are prevented during their use. Do not use reagents over the expiration date.

Reagent deterioration: The presence of particles and turbidity.

Do not freeze; frozen Antibody or Diluent could change the functionality of the test.

ADDITIONAL EQUIPMENT

- SPIN640 / SPIN640Plus autoanalyzer
- Laboratory equipment.

SAMPLES

Fresh serum or plasma. EDTA or heparin should be used as anticoagulant. Stable 7 days at 2-8°C or 3 months at -20°C. Do not use highly hemolyzed or lipemic samples.

REFERENCE VALUES

Between 50 - 120 mg/dL. Each laboratory should establish its own reference range.

QUALITY CONTROL

Control sera are recommended to monitor the performance of manual and automated assay procedures. Spinreact PROT CONTROL (Ref.:1102004) is available. Each laboratory should establish its own Quality Control scheme and corrective actions if controls do not meet the acceptable tolerances.

NOTES

Clinical diagnosis should not be made on findings of a single test result, but should integrate both clinical and laboratory data.

BARCODED REAGENTS LOAD MUST BE PRECEDED OF A SPINREACT "DATABASE" COPY INTO THE ANALYZER SOFTWARE. IT IS AVAILABLE UNDER REQUEST TO SPINREACT.

SPIN640 APPLICATION

TEST INFORMATION		REAGENT VOLUME		
Nº	**	Vol. R1	240	
Test	AcGLY	Vol. R2	60	
Full Name	Alpha1AcidGlicoprot	Vol. R3		
Standard nº	6	Vol. R4		
SAMPLE VOLUME		RESULT SETUP		
Vol. Sample Stand.	3	Decimal	1 Slope 1	
Vol. Sample Increas.		Unit	mg/dL Inter. 0	
Vol. Sample Dec				
REACTION PARAMETERS				
Reac. Type	End Point	Direction	Increase	
Pri. Wave.	340	Reagent Blank	20-21	
Sec. Wave.		React. Time	56-57	
CALIBRATION DILUTION (5 CAL + 1 WATER BLANK)				
CALIBRATION TYPE: SPLINE				
Nº CAL DIL	CONCENTRATION	SAMPLE DIL	VOL DIL	VOL SAMPLE
0	CAL *0.1	10.0	90.0	3.0
1	CAL *0.25	30.0	90.0	3.0
2	CAL *0.4951	30.0	90.0	6.0
3	CAL *0.7354	30.0	90.0	9.0
4	CAL	0.0	0.0	3.0

SPIN640Plus APPLICATION

EDIT PARAMETERS			
Test	AcGLY	No.	**
Full name	AcGLY	Print name	AcGLY
Reac. Type	End Point	Direction	Increase
Pri. Wave.	340	Sec. Wave.	
Unit	mg/dL	Decimal	0.1
Reagent Blank	20-21	React. Time	63-64
Vol. Sample	3 ul	R1	240 ul
Increased		R2	60 ul
Decreased		R3	
Sample blank		R4	
CALIBRATION	Spline	Water	0
Calibrator	*0.1	Vol aspirated	3 13 115
Calibrator	*0.25	Vol net	3 35 105
Calibrator	*0.489	Vol dil	6 35 105
Calibrator	*0.735		9 35 105
Calibrator	(*)Cal Value		3 -- --

The Calibration is stable until 31 days. After this period the Calibration must be performed again in order to obtain good results.

PERFORMANCE CHARACTERISTICS

- Measurement range:** Up to 250 mg/dL under the described assay conditions. Samples with higher concentrations, should be diluted 1/5 in NaCl 9 g/L and retested again. The linearity limit depends on the sample / reagent ratio. It will be higher by decreasing the sample volume, although the sensitivity of the test will be proportionally decreased.
- Detection Limit:** Values less than 12,9 mg/dL give non-reproducible results.
- Prozone effect:** No prozone effect was detected upon 1000 mg/dL.
- Sensitivity:** Δ 5.0 mA / mg/dL.
- Precision:** The reagent has been tested for 20 days, using two levels of serum in a EP5-based study.

EP5	CV (%)	
	56,4 mg/dL	112,07 mg/dL
Total	3,3%	3,1%
Within Run	1,1%	1,6%
Between Run	3%	2,1%
Between Day	0,7%	1,6%

- Accuracy:** Results obtained using this reagent (y) were compared to those obtained using the method Immage from Beckman. 51 samples ranging from 50 to 120 mg/dL of α₁-Ac Gly were assayed. The correlation coefficient (r) was 0.95 and the regression equation $y = 0,9304x + 6,5367$.

The results of the performance characteristics depend on the used analyzer.

BIBLIOGRAPHY

- Clinical Guide to Laboratory Tests, Edited by NW Tietz W B Saunders Co., Philadelphia, 483, 1983.
- Dati F et al. Eur J Clin Chem Clin Biochem 1996; 34:517-520.
- Pesce AJ and Kaplan, LA. Methods in Clinical Chemistry. The CV Mosby Company, St. Louis MO, 1987.
- Bienvenue J et al. Clin Chem Clin Biochem 1981; 27: 721-726.
- Young DS. Effects of drugs on clinical laboratory tests, 4th ed. AACCC Pres, 1995.
- Friedman and Young. Effects of disease on clinical laboratory tests, 3th ed. AACCC Pres, 1997.

PACKAGING

Ref: MD1102154	Cont.	R1.Diluent:	2 x 40 mL
		R2.Antibody:	1 x 20 mL

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EDIT PARAMETERS																																													
Test	AcGLY	No.	**																																										
Full name	AcGLY	Print name	AcGLY																																										
Reac. Type	End Point	Direction	Increase																																										
Pri. Wave.	340	Sec. Wave.																																											
Unit	mg/dL	Decimal	0.1																																										
Reagent Blank	20-21	React. Time	63 - 64																																										
Vol. Sample	3 ul	R1	240 ul																																										
Increased		R2	60 ul																																										
Decreased		R3																																											
Sample blank		R4																																											
<table border="1"> <thead> <tr> <th>CALIBRATION</th> <th>Spline</th> <th>Water</th> <th>0</th> <th>Vol aspirated</th> <th>Vol net</th> <th>Vol dil</th> </tr> </thead> <tbody> <tr> <td>Calibrator</td> <td>*0.1</td> <td></td> <td></td> <td>3</td> <td>13</td> <td>115</td> </tr> <tr> <td>Calibrator</td> <td>*0.25</td> <td></td> <td></td> <td>3</td> <td>35</td> <td>105</td> </tr> <tr> <td>Calibrator</td> <td>*0.489</td> <td></td> <td></td> <td>6</td> <td>35</td> <td>105</td> </tr> <tr> <td>Calibrator</td> <td>*0.735</td> <td></td> <td></td> <td>9</td> <td>35</td> <td>105</td> </tr> <tr> <td>Calibrator</td> <td>(*)Cal Value</td> <td></td> <td></td> <td>3</td> <td>--</td> <td>--</td> </tr> </tbody> </table>				CALIBRATION	Spline	Water	0	Vol aspirated	Vol net	Vol dil	Calibrator	*0.1			3	13	115	Calibrator	*0.25			3	35	105	Calibrator	*0.489			6	35	105	Calibrator	*0.735			9	35	105	Calibrator	(*)Cal Value			3	--	--
CALIBRATION	Spline	Water	0	Vol aspirated	Vol net	Vol dil																																							
Calibrator	*0.1			3	13	115																																							
Calibrator	*0.25			3	35	105																																							
Calibrator	*0.489			6	35	105																																							
Calibrator	*0.735			9	35	105																																							
Calibrator	(*)Cal Value			3	--	--																																							

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BIBLIOGRAFIA

- Clinical Guide to Laboratory Tests, Edited by NW Tietz W B Saunders Co., Philadelphia, 483, 1983.
- Dati F et al. Eur J Clin Chem Clin Biochem 1996; 34:517-520.
- Pesce AJ and Kaplan, LA. Methods in Clinical Chemistry. The CV Mosby Company, St. Louis MO, 1987.
- Bienvue J et al. Clin Chem Clin Biochem 1981; 27: 721-726.
- Young DS. Effects of drugs on clinical laboratory tests, 4th ed. AACC Pres, 1995.
- Friedman and Young. Effects of disease on clinical laboratory tests, 3th ed. AACC Pres, 1997.

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TEST INFORMATION		REAGENT VOLUME		
N°	**	Vol. R1	240	
Test	AcGLY	Vol. R2	60	
Full Name	Alpha1Antytripsin	Vol. R3		
Standard n°	6	Vol. R4		
SAMPLE VOLUME		RESULT SETUP		
Vol. Sample Stand.	3	Decimal	1 Slope 1	
Vol. Sample Increas.		Unit	mg/dL Inter. 0	
Vol. Sample Dec				
REACTION PARAMETERS				
Reac. Type	End Point	Direction	Increase	
Pri. Wave.	340	Reagent Blank	20-21	
Sec. Wave.		React. Time	56-57	
CALIBRATION DILUTION (5 CAL + 1 WATER BLANK)				
CALIBRATION TYPE: SPLINE				
N° CAL DIL	CONCENTRATION	SAMPLE DIL	VOL DIL	VOL SAMPLE
0	CAL *0,1	10,0	90,0	3,0
1	CAL *0,25	30,0	90,0	3,0
2	CAL *0,4951	30,0	90,0	6,0
3	CAL *0,7354	30,0	90,0	9,0
4	CAL	0,0	0,0	3,0

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BIBLIOGRAPHIE

- Clinical Guide to Laboratory Tests, Edited by NW Tietz W B Saunders Co., Philadelphia, 483, 1983.
- Dati F et al. Eur J Clin Chem Clin Biochem 1996; 34:517-520.
- Pesce AJ and Kaplan, LA. Methods in Clinical Chemistry. The CV Mosby Company, St. Louis MO, 1987.
- Bienvenue J et al. Clin Chem Clin Biochem 1981; 27: 721-726.
- Young DS. Effects of drugs on clinical laboratory tests, 4th ed. AACC Pres, 1995.
- Friedman and Young. Effects of disease on clinical laboratory tests, 3th ed. AACC Pres, 1997.

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BIBLIOGRAFIA

- Clinical Guide to Laboratory Tests, Edited by NW Tietz W B Saunders Co., Philadelphia, 483, 1983.
- Dati F et al. Eur J Clin Chem Clin Biochem 1996; 34:517-520.
- Pesce AJ and Kaplan, LA. Methods in Clinical Chemistry. The CV Mosby Company, St. Louis MO, 1987.
- Bienvenue J et al. Clin Chem Clin Biochem 1981; 27: 721-726.
- Young DS. Effects of drugs on clinical laboratory tests, 4th ed. AACCPres, 1995.
- Friedman and Young. Effects of disease on clinical laboratory tests, 3th ed. AACCPres, 1997.

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Ref.: MD1102154

Cont.

 R1. Solvente: 2 x 40 ml
 R2. Anticorpo: 1 x 20 ml