Quantitative determination of Immunoglobulin A (IgA) IVD

Store at 2-8°C

PRINCIPLE OF THE METHOD

Anti-human IgA antibodies when mixed with samples containing IgA, form insoluble complexes. These complexes cause an absorbance change, dependent upon the IgA concentration of the patient sample, that can be quantified by comparison from a calibrator of known IgA concentration.

CLINICAL SIGNIFICANCE

IgA represents approximately 10 to 15% of total serum immunoglobulins. Its structure is monomeric, similar to the IgG molecule, but 10 to 15% of IgA in serum is polymeric, particularly IgA₂, which is more resistant to destruction by some pathogenic bacteria. Another more important form of IgA is called secretory IgA. It is found in tears, sweat, saliva, milk and gastrointestinal and bronchial secretions.

IgA is generally increased in skin, pulmonary, kidney infections, and hepatic cirrhosis. Increased monoclonal IgA concentrations may be found in multiple myeloma and other disturbances of plasmatic cells.

REAGENTS

R 1 Diluent	Tris buffer 20 mmol/L, PEG 8000, pH, 8,3. Preservative.
R 2 Antibody	Goat serum, anti-human IgA, pH 7,5. Preservative.
Optional	Ref: 1102003 PROT CAL.

CALIBRATION

The assay is calibrated to the Reference Material ERM-DA470k/IFCC. It must be used the PROT CAL Calibrator to calibrate the reagent.

PREPARATION

Reagents: Ready to use.

STORAGE AND STABILITY

All the components of the kit are stable until the expiration date on the label when stored tightly closed at 2-8°C and contaminations are prevented during their use. Do not use reagents over the expiration date.

Reagent deterioration: The presence of particles and turbidity.

Do not freeze; frozen Antibody or Diluent could change the functionality of the test.

ADDITIONAL EQUIPMENT

- SPIN640 / SPIN640Plus autoanalyzer
- Laboratory equipment.

SAMPLES

Fresh serum or plasma. EDTA or heparin should be used as anticoagulant. Stable 7 days at 2-8°C or 3 months at -20°C.

The samples with presence of fibrin should be centrifuged.

Do not use highly hemolyzed or lipemic samples.

REFERENCE VALUES

Between 70 - 400 mg/dL. Each laboratory should establish its own reference range.

QUALITY CONTROL

Control sera are recommended to monitor the performance of manual and automated assay procedures. Spinreact PROT CONTROL (Ref.:1102004) is available. Each laboratory should establish its own Quality Control scheme and corrective actions if controls do not meet the acceptable tolerances.

NOTES

Clinical diagnosis should not be made on findings of a single test result, but should integrate both clinical and laboratory data.

BARCODED REAGENTS LOAD MUST BE PRECEDED OF A SPINREACT "DATABASE" COPY INTO THE ANALYZER SOFTWARE. IT IS AVAILABLE UNDER REQUEST TO SPINREACT.

SPIN640 APPLICATION

TEST INFORMATION		REAGENT VOLUME		
Nº	**	Vol. R1	240	
Test	IgA	Vol. R2	60	
Full Name	Immunoglobulin A	Vol. R3		
Standard nº	6	Vol. R4		
SAMPLE VOLUME		RESULT SETUP		
Vol. Sample Stand.	3	Decimal	0.1 Slope 1	
Vol. Sample Inceas.		Unit	mg/dL Inter. 0	
Vol. Sample Dec				
REACTION PARAMETERS				
Reac. Type	End Point	Direction	Increase	
Pri. Wave.	605	Reagent Blank	40-41	
Sec. Wave.		React. Time	65-66	
CALIBRATION DILUTION (5 CAL + 1 WATER BLANK)				
CALIBRATION TYPE: SPLINE				
Nº CAL DIL	CONCENTRATION	SAMPLE DIL	VOL DIL	VOL SAMPLE
0	CAL *0.1	10.0	90.0	3.0
1	CAL *0.25	30.0	90.0	3.0
2	CAL *0.4951	30.0	90.0	6.0
3	CAL *0.7356	30.0	90.0	9.0
4	CAL	0.0	0.0	3.0

SPIN640Plus APPLICATION

EDIT PARAMETERS			
Test	IgA	No.	**
Full name	Immunoglobulin A	Print name	IgA
Reac. Type	End Point	Direction	Increase
Pri. Wave.	605	Sec. Wave.	
Unit	mg/dL	Decimal	0.1
Reagent Blank	46-47	React. Time	72 - 73
Vol. Sample	3 ul	R1	240 ul
Increased		R2	60 ul
Decreased		R3	
Sample blank		R4	
CALIBRATION	Spline	Water	0
	Calibrator	*0.1	3 13 115
	Calibrator	*0.25	3 35 105
	Calibrator	*0.489	6 35 105
	Calibrator	*0.735	9 35 105
	Calibrator	(*Cal Value	3 -- --

The Calibration is stable until **31 days**. After this period the Calibration must be performed again in order to obtain good results.

PERFORMANCE CHARACTERISTICS

- Measurement range: Up to 600 mg/dL, under the described assay conditions. Samples with higher concentrations, should be diluted 1/5 in NaCl 9 g/L and re-tested again. The linearity limit and measurement range depends on the sample to reagent / ratio. It will be higher by decreasing the sample volume, although the sensitivity of the test will be proportionally decreased.
- Limit detection: Values less than 0,0006 mg/dL give non-reproducible results.
- Prozone effect: No prozone effect was detected upon 2000 mg/dL
- Sensitivity: Δ 2,1 mA. mg/dL at 71 mg/dL.
- Precision: The reagent has been tested for 20 days, using three levels of serum in a EP5-based study.

EP5	CV (%)		
	127,7 mg/dL	196,9 mg/dL	416,3 mg/dL
Total	8,2%	5,2%	3,5%
Within Run	1,7%	1,5%	1%
Between Run	2,2%	1,9%	2,4%
Between Day	7,7%	4,6%	2,3%

- Accuracy: Results obtained using this reagent (y) were compared to those obtained using an immunoturbidimetric method from Bayer. 46 samples ranging from 20 to 400 mg/dL of IgA were assayed. The correlation coefficient (r) was 0,97 and the regression equation $y = 1,16x - 12,2$.

The results of the performance characteristics depend on the used analyzer.

BIBLIOGRAPHY

- Clinical Guide to Laboratory Tests, Edited by NW Tietz W B Saunders Co., Philadelphia, 483, 1983.
- Skoug Jonh W et al. Clin Chem 1988; 34/2: 309 - 315
- Pesce AJ and Kaplan, LA. Methods in Clinical Chemistry. The CV Mosby Company, St. Louis MO, 1987.
- Dati F et al. Eur J Clin Chem Clin Biochem 1996; 34: 517-520.
- Young DS. Effects of disease on clinical laboratory tests, 3th ed. AACC Pres, 1997
- Friedman and Young. Effects of disease on clinical laboratory tests, 3th ed. AACC Pres, 1997.

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TEST INFORMATION		REAGENT VOLUME		
N°	**	Vol. R1	240	
Test	IgA	Vol. R2	60	
Full Name	ImmunoglobulinA	Vol. R3		
Standard n°	6	Vol. R4		
SAMPLE VOLUME		RESULT SETUP		
Vol. Sample Stand.	3	Decimal	0.1 Slope 1	
Vol. Sample Increas.		Unit	mg/dL Inter. 0	
Vol. Sample Dec				
REACTION PARAMETERS				
Reac. Type	End Point	Direction	Increase	
Pri. Wave.	605	Reagent Blank	40-41	
Sec. Wave.		React. Time	65-66	
CALIBRATION DILUTION (5 CAL + 1 WATER BLANK)				
CALIBRATION TYPE: SPLINE				
N° CAL DIL	CONCENTRATION	SAMPLE DIL	VOL DIL	VOL SAMPLE
0	CAL *0.1	10.0	90.0	3.0
1	CAL *0.25	30.0	90.0	3.0
2	CAL *0.4951	30.0	90.0	6.0
3	CAL *0.7356	30.0	90.0	9.0
4	CAL	0.0	0.0	3.0

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BIBLIOGRAPHIE

- Clinical Guide to Laboratory Tests, Edited by NW Tietz W B Saunders Co., Philadelphia, 483, 1983.
- Skoug Jonh W et al. Clin Chem 1988; 34/2: 309 - 315
- Pesce AJ and Kaplan, LA. Methods in Clinical Chemistry. The CV Mosby Company, St. Louis MO, 1987.
- Dati F et al. Eur J Clin Chem Clin Biochem 1996; 34: 517-520.
- Young DS. Effects of disease on clinical laboratory tests, 3th ed. AACC Pres, 1997.
- Friedman and Young. Effects of disease on clinical laboratory tests, 3th ed. AACC Pres, 19

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BIBLIOGRAFIA

1. Clinical Guide to Laboratory Tests, Edited by NW Tietz W B Saunders Co., Philadelphia, 483, 1983.
2. Skoug Jonh W et al. Clin Chem 1988; 34/2: 309 - 315
3. Pesce AJ and Kaplan, LA. Methods in Clinical Chemistry. The CV Mosby Company, St. Louis MO, 1987.
4. Dati F et al. Eur J Clin Chem Clin Biochem 1996; 34: 517-520.
5. Young DS. Effects of disease on clinical laboratory tests, 3th ed. AACC Pres, 1997
6. Friedman and Young. Effects of disease on clinical laboratory tests, 3th ed. AACC Pres, 1997.

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Ref: MD1103014

Cont.

R1.Solvente: 2 x 40 mL

R2.Anticorpo: 1 x 20 mL