Quantitative determination of apolipoprotein A-I (APO A-I) IVD

Store 2 - 8°C.

PRINCIPLE OF THE METHOD

Turbidimetric test for the measurement of apolipoprotein A-I in human serum or plasma.

Anti- Apo A-I antibodies when mixed with samples containing Apo A-I, form insoluble complexes. These complexes cause an absorbance change, dependent upon the Apo A-I concentration of the patient sample, that can be quantified by comparison from a calibrator of known Apo A-I concentration.

CLINICAL SIGNIFICANCE¹

Apo A-I is the major structural apolipoprotein in HDL and constitutes about 70% of the total protein. Apo A-I is a cofactor for lecithin-cholesterol-acyl-transferase (LCAT), the enzyme responsible for forming cholesteryl esters in plasma and plays an important role in the transport of cholesterol from peripheral tissues to the liver, to be finally excreted. Measurements of Apo A-I concentration is especially important in detecting coronary heart disease risk (CHD) as well as in the diagnostic of hyperlipoproteinemia. Concentrations < 120 mg/L are associated to an increased CHD risk, while concentrations ≥ 160 mg/L may even protect from the same risk. Patients with deficiencies in Apo A-I synthesis may highly increase the CHD risk.

Tanger disease, a consequence of an Apo A-I catabolism defect, is characterized by several reduced plasma HDL cholesterol (HDL-c) concentration, abnormal HDL composition and accumulation of cholesteryl esters in many body tissues. Plasma HDL-c and Apo A-I concentrations in homozygotes are very low, while Apo A-I concentration is less than 10% of its normal concentration. Heterozygotes are characterized by half-normal concentration of HDL-c, Apo A-I and Apo -II. Current evidence suggests that these patients have increased incidence of CHD.

REAGENTS

Diluent (R1)	Tris buffer 20 mmol/L, PEG, pH 8.3. Preservative.
Antibody (R2)	Goat serum, anti-human Apo B, tris 50 mmol/L, pH 7.5. Preservative.
Optional	APO CAL ref: 93005

CALIBRATION

The assay and the value of the calibrator concentration have been standardized against the Certified Reference Material WHO/IFCC SP1-01 (CDC, USA). It is recommended the use of the APO CAL Calibrator for calibration.

PREPARATION

Reagents: Ready to use.

STORAGE AND STABILITY

All the components of the kit are stable until the expiration date on the label when stored tightly closed at 2-8°C and contaminations are prevented during their use. Do not use reagents over the expiration date.

Reagent deterioration: The presence of particles and turbidity.

Do not freeze; frozen Antibody or Diluent could change the functionality of the test.

ADDITIONAL EQUIPMENT

- SPIN640 / SPIN640Plus autoanalyzer
- Laboratory equipment.

SAMPLES

Fresh serum or plasma. EDTA or heparin should be used as anticoagulant. Stable 2 weeks at 2-8°C or 3 months at -20°C.

The samples with presence of fibrin should be centrifuged before testing.

Do not use highly hemolyzed or lipemic samples.

REFERENCE VALUES⁵

Between 122 – 161 mg/dL.

Each laboratory should establish its own reference range.

QUALITY CONTROL

Control sera are recommended to monitor the performance of manual and automated assay procedures. Spinreact Apolipoprotein Control (Ref.:93006) is available. Each laboratory should establish its own Quality Control scheme and corrective actions if controls do not meet the acceptable tolerances.

NOTES

Clinical diagnosis should not be made on findings of a single test result, but should integrate both clinical and laboratory data.

BARCODED REAGENTS LOAD MUST BE PRECEDED OF A SPINREACT "DATABASE" COPY INTO THE ANALYZER SOFTWARE. IT IS AVAILABLE UNDER REQUEST TO SPINREACT.

SPIN640 APPLICATION

TEST INFORMATION		REAGENT VOLUME		
Nº	**	Vol. R1	240	
Test	ApoA1	Vol. R2	60	
Full Name	ApolipoproteinA1	Vol. R3		
Standard nº	6	Vol. R4		
SAMPLE VOLUME		RESULT SETUP		
Vol. Sample Stand.	2	Decimal	1 Slope 1	
Vol. Sample Incr.		Unit	mg/dL Inter. 0	
Vol. Sample Dec				
REACTION PARAMETERS				
Reac. Type	Fixed Time	Direction	Increase	
Pri. Wave.	340	Reagent Blank	0-0	
Sec. Wave.		React. Time	43-76	
CALIBRATION DILUTION (5 CAL + 1 WATER BLANK)				
CALIBRATION TYPE: SPLINE				
Nº CAL DIL	CONCENTRATION	SAMPLE DIL	VOL DIL	VOL SAMPLE
0	CAL *0.1	10.0	90.0	2.0
1	CAL *0.25	30.0	90.0	2.0
2	CAL *0.4971	30.0	90.0	4.0
3	CAL *0.7412	30.0	90.0	6.0
4	CAL	0.0	0.0	2.0

SPIN640Plus APPLICATION

EDIT PARAMETERS																																													
Test	APO A1	No.	**																																										
Full name	APO A1	Print name	APO A1																																										
Reac. Type	Fixed Time	Direction	Increase																																										
Pri. Wave.	340	Sec. Wave.																																											
Unit	mg/dL	Decimal	0.1																																										
Reagent Blank	15 - 16	React. Time	51 - 82																																										
Vol. Sample	2.1 ul	R1	240 ul																																										
Increased		R2	60 ul																																										
Decreased		R3																																											
Sample blank		R4																																											
<table border="1"> <thead> <tr> <th>CALIBRATION</th> <th>Spline</th> <th>Water</th> <th>0</th> <th>Vol aspirated</th> <th>Vol net</th> <th>Vol dil</th> </tr> </thead> <tbody> <tr> <td>Calibrator</td> <td>*0.1</td> <td></td> <td></td> <td>2.1</td> <td>13</td> <td>115</td> </tr> <tr> <td>Calibrator</td> <td>*0.25</td> <td></td> <td></td> <td>2.1</td> <td>35</td> <td>105</td> </tr> <tr> <td>Calibrator</td> <td>*0.489</td> <td></td> <td></td> <td>4.2</td> <td>35</td> <td>105</td> </tr> <tr> <td>Calibrator</td> <td>*0.735</td> <td></td> <td></td> <td>6.3</td> <td>35</td> <td>105</td> </tr> <tr> <td>Calibrator</td> <td>(*)Cal Value</td> <td></td> <td></td> <td>2.1</td> <td>--</td> <td>--</td> </tr> </tbody> </table>				CALIBRATION	Spline	Water	0	Vol aspirated	Vol net	Vol dil	Calibrator	*0.1			2.1	13	115	Calibrator	*0.25			2.1	35	105	Calibrator	*0.489			4.2	35	105	Calibrator	*0.735			6.3	35	105	Calibrator	(*)Cal Value			2.1	--	--
CALIBRATION	Spline	Water	0	Vol aspirated	Vol net	Vol dil																																							
Calibrator	*0.1			2.1	13	115																																							
Calibrator	*0.25			2.1	35	105																																							
Calibrator	*0.489			4.2	35	105																																							
Calibrator	*0.735			6.3	35	105																																							
Calibrator	(*)Cal Value			2.1	--	--																																							

The Calibration is stable until **27 days**. After this period the Calibration must be performed again in order to obtain good results.

PERFORMANCE CHARACTERISTICS

- Measurement range:** Up to 250 mg/dL, under the described assay conditions. Samples with higher concentrations, should be diluted 1/5 in NaCl 9 g/L and retested again. The linearity limit depends on the sample / reagent ratio. It will be higher by decreasing the sample volume, although the sensitivity of the test will be proportionally decreased.
- Detection Limit:** Values less than 0,1 mg/dL give non-reproducible results.
- Precision:** The reagent has been tested for 20 days, using three levels of serum in a EP5-based study (NCCLS).

EP5	CV (%)		
	27,22mg/dL	65,74 mg/dL	131,07 mg/dL
Total	4%	3,7%	4,8%
Within Run	2,2%	0,8%	1,1%
Between Run	2,3%	1,3%	1,4%
Between Day	2,4%	3,3%	4,5%

- Accuracy:** Results obtained using this reagent (y) were compared to those obtained with a Bayer immunoturbidimetric method. 39 samples ranging from 50 to 200 mg/dL of Apo A-I were assayed. The correlation coefficient (r) was 0,92 and the regression equation $y = 1,18x - 37,8$.

The results of the performance characteristics depend on the used analyzer.

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TEST INFORMATION		REAGENT VOLUME		
Nº	**	Vol. R1	240	
Test	ApoA1	Vol. R2	60	
Full Name	ApolipoproteinA1	Vol. R3		
Standard nº	6	Vol. R4		
SAMPLE VOLUME		RESULT SETUP		
Vol. Sample Stand.	2	Decimal	1 Slope 1	
Vol. Sample Inceas.		Unit	mg/dL Inter. 0	
Vol. Sample Dec				
REACTION PARAMETERS				
Reac. Type	Fixed Time	Direction	Increase	
Pri. Wave.	340	Reagent Blank	0-0	
Sec. Wave.		React. Time	43-76	
CALIBRATION DILUTION (5 CAL + 1 WATER BLANK)				
CALIBRATION TYPE: SPLINE				
Nº CAL DIL	CONCENTRATION	SAMPLE DIL	VOL DIL	VOL SAMPLE
0	CAL *0.1	10.0	90.0	2.0
1	CAL *0.25	30.0	90.0	2.0
2	CAL *0.4971	30.0	90.0	4.0
3	CAL *0.7412	30.0	90.0	6.0
4	CAL	0.0	0.0	2.0

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EDIT PARAMETERS			
Test	APO A1	No.	**
Full name	APO A1	Print name	APO A1
Reac. Type	Fixed Time	Direction	Increase
Pri. Wave.	340	Sec. Wave.	
Unit	mg/dL	Decimal	0.1
Reagent Blank	15 - 16	React. Time	51 - 82
Vol. Sample	2.1 ul	R1	240 ul
Increased		R2	60 ul
Decreased		R3	
Sample blank		R4	
CALIBRATION			
Spline	Water	0	
Calibrator	*0.1	2.1	13 115
Calibrator	*0.25	2.1	35 105
Calibrator	*0.489	4.2	35 105
Calibrator	*0.735	6.3	35 105
Calibrator	(*)Cal Value	2.1	-- --

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APPLICATION AU SPIN640

TEST INFORMATION		REAGENT VOLUME		
N°	**	Vol. R1	240	
Test	ApoA1	Vol. R2	60	
Full Name	ApolipoproteinA1	Vol. R3		
Standard n°	6	Vol. R4		
SAMPLE VOLUME		RESULT SETUP		
Vol. Sample Stand.	2	Decimal	1 Slope 1	
Vol. Sample Increas.		Unit	mg/dL Inter. 0	
Vol. Sample Dec				
REACTION PARAMETERS				
Reac. Type	Fixed Time	Direction	Increase	
Pri. Wave.	340	Reagent Blank	0-0	
Sec. Wave.		React. Time	43-76	
CALIBRATION DILUTION (5 CAL + 1 WATER BLANK)				
CALIBRATION TYPE: SPLINE				
N° CAL DIL	CONCENTRATION	SAMPLE DIL	VOL DIL	VOL SAMPLE
0	CAL *0.1	10.0	90.0	2.0
1	CAL *0.25	30.0	90.0	2.0
2	CAL *0.4971	30.0	90.0	4.0
3	CAL *0.7412	30.0	90.0	6.0
4	CAL	0.0	0.0	2.0

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TEST INFORMATION		REAGENT VOLUME		
Nº	**	Vol. R1	240	
Test	ApoA1	Vol. R2	60	
Full Name	ApolipoproteinA1	Vol. R3		
Standard nº	6	Vol. R4		
SAMPLE VOLUME		RESULT SETUP		
Vol. Sample Stand.	2	Decimal	1 Slope 1	
Vol. Sample Increas.		Unit	mg/dL Inter. 0	
Vol. Sample Dec				
REACTION PARAMETERS				
Reac. Type	Fixed Time	Direction	Increase	
Pri. Wave.	340	Reagent Blank	0-0	
Sec. Wave.		React. Time	43-76	
CALIBRATION DILUTION (5 CAL + 1 WATER BLANK)				
CALIBRATION TYPE: SPLINE				
Nº CAL DIL	CONCENTRATION	SAMPLE DIL	VOL DIL	VOL SAMPLE
0	CAL *0.1	10.0	90.0	2.0
1	CAL *0.25	30.0	90.0	2.0
2	CAL *0.4971	30.0	90.0	4.0
3	CAL *0.7412	30.0	90.0	6.0
4	CAL	0.0	0.0	2.0

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EDIT PARAMETERS				
Test	APO A1	No.	**	
Full name	APO A1	Print name	APO A1	
Reac. Type	Fixed Time	Direction	Increase	
Pri. Wave.	340	Sec. Wave.		
Unit	mg/dL	Decimal	0.1	
Reagent Blank	15 - 16	React. Time	51 - 82	
Vol. Sample	2.1 ul	R1	240 ul	
Increased		R2	60 ul	
Decreased		R3		
Sample blank		R4		
CALIBRATION Spline Water 0				
Calibrator	*0.1	Vol aspirated	Vol net	Vol dil
Calibrator	*0.25	2.1	13	115
Calibrator	*0.489	2.1	35	105
Calibrator	*0.735	4.2	35	105
Calibrator	(*)Cal Value	6.3	35	105
		2.1	--	--

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