APO-B

# Apolipoprotein B

Turbidimetry

## Quantitative determination of apolipoprotein B (APO B) IVD

Store 2 - 8°C.

### PRINCIPLE OF THE METHOD

Turbidimetric test for the measurement of apolipoprotein B in human serum or plasma.

Anti- Apo B antibodies when mixed with samples containing Apo B, form insoluble complexes. These complexes cause an absorbance change, dependent upon the Apo B concentration of the patient sample, that can be quantified by comparison from a calibrator of known Apo B concentration.

### CLINICAL SIGNIFICANCE<sup>1</sup>

Apo B is the major structural apolipoprotein in VLDL (Very Low Density Lipids), LDL (Low Density Lipids) lipoproteins and chylomicron. The most important function is the transport of rich tryglicerides lipoproteins from liver and intestine to other tissues. Apo B exists in two forms: Apo B-100 and Apo B-48. Apo B-100, the most important component of LDL, is synthesized in the liver and excreted in plasma as part of VLDL. Apo B-48, the most important component of chylomicrons, is synthesized in the intestine.

Several studies demonstrated that in people with coronary heart disease (CHD), changes in the serum concentrations of Apo A-I and Apo B are similar to those for HDL and LDL, respectively and whereas, are somewhat better discriminators of people with CHD than the LDL and HDL cholesterol concentrations.

The hiperbetalipoproteinemia is characterized by increased LDL Apo B-100 concentrations with normal or moderately increased concentrations of LDL cholesterol. The ratio of LDL cholesterol to Apo B-100 is therefore reduced in these patients.

Defects in the Apo B structure or lipoproteins containing Apo B prevent the secretion of triglycerides rich intestinal and hepatic lipoproteins; this disorder occurs in abetalipoproteinemia or homozygous hypobetalipoproteinemia.

### REAGENTS

<b>Diluent (R1)</b>	Tris buffer 20 mmol/L, PEG, pH 8,3. Preservative.
<b>Antibody (R2)</b>	Goat serum, anti-human Apo B, tris 50 mmol/L, pH 7,5. Preservative.
<b>Optional</b>	APO CAL ref: 93005

### CALIBRATION

The assay and the value of the calibrator concentration have been standardized against the Certified Reference Material WHO/IFCC SP3-07 (CDC, USA). It is recommended the use of the APO CAL Calibrator for calibration.

### PREPARATION

**Reagents:** Ready to use.

### STORAGE AND STABILITY

All the components of the kit are stable until the expiration date on the label when stored tightly closed at 2-8°C and contaminations are prevented during their use. Do not use reagents over the expiration date.

**Reagent deterioration:** The presence of particles and turbidity.

Do not freeze; frozen Antibody or Diluent could change the functionality of the test.

### ADDITIONAL EQUIPMENT

- SPIN640 / SPIN640Plus autoanalyzer
- Laboratory equipment.

### SAMPLES

Fresh serum or plasma. EDTA or heparin should be used as anticoagulant. Stable 2 weeks at 2-8°C or 3 months at -20°C.

The samples with presence of fibrin should be centrifuged before testing.

Do not use highly hemolyzed or lipemic samples.

### REFERENCE VALUES<sup>5</sup>

Between 69 – 105 mg/dL.

Each laboratory should establish its own reference range.

### QUALITY CONTROL

Control sera are recommended to monitor the performance of manual and automated assay procedures. Spinreact Apolipoprotein Control (Ref.:93006) is available. Each laboratory should establish its own Quality Control scheme and corrective actions if controls do not meet the acceptable tolerances.

### NOTES

Clinical diagnosis should not be made on findings of a single test result, but should integrate both clinical and laboratory data.

**BARCODED REAGENTS LOAD MUST BE PRECEDED OF A SPINREACT "DATABASE" COPY INTO THE ANALYZER SOFTWARE. IT IS AVAILABLE UNDER REQUEST TO SPINREACT.**

### SPIN640 APPLICATION

<b>TEST INFORMATION</b>		<b>REAGENT VOLUME</b>		
Nº	**	Vol. R1	240	
Test	ApoB	Vol. R2	60	
Full Name	ApolipoproteinB	Vol. R3		
Standard nº	6	Vol. R4		
<b>SAMPLE VOLUME</b>		<b>RESULT SETUP</b>		
Vol. Sample Stand.	2	Decimal	1 Slope 1	
Vol. Sample Increas.		Unit	mg/dL Inter. 0	
Vol. Sample Dec				
<b>REACTION PARAMETERS</b>				
Reac. Type	End Point	Direction	Increase	
Pri. Wave.	340	Reagent Blank	40-41	
Sec. Wave.		React. Time	75-76	
<b>CALIBRATION DILUTION (5 CAL + 1 WATER BLANK)</b>				
CALIBRATION TYPE: SPLINE				
<b>Nº CAL DIL</b>	<b>CONCENTRATION</b>	<b>SAMPLE DIL</b>	<b>VOL DIL</b>	<b>VOL SAMPLE</b>
0	CAL *0.1	10.0	90.0	2.0
1	CAL *0.25	30.0	90.0	2.0
2	CAL *0.4971	30.0	90.0	4.0
3	CAL *0.7412	30.0	90.0	6.0
4	CAL	0.0	0.0	2.0

### SPIN640Plus APPLICATION

<b>EDIT PARAMETERS</b>																												
Test	APO B	No.	**																									
Full name	APO B	Print name	APO B																									
Reac. Type	End Point	Direction	Increase																									
Pri. Wave.	340	Sec. Wave.																										
Unit	mg/dL	Decimal	0.1																									
Reagent Blank	46 - 47	React. Time	81 - 82																									
Vol. Sample	2.1 ul	R1	240 ul																									
Increased		R2	60 ul																									
Decreased		R3																										
Sample blank		R4																										
<table border="1"> <tr> <td>CALIBRATION</td> <td>Spline</td> <td>Water</td> <td>0</td> <td rowspan="6"> <i>Vol aspirated</i>   <i>Vol net</i>   <i>Vol dil</i>  2.1   13   115  2.1   35   105  4.2   35   105  6.3   35   105  2.1   --   -- </td> </tr> <tr> <td></td> <td>Calibrator</td> <td>*0.1</td> <td></td> </tr> <tr> <td></td> <td>Calibrator</td> <td>*0.25</td> <td></td> </tr> <tr> <td></td> <td>Calibrator</td> <td>*0.489</td> <td></td> </tr> <tr> <td></td> <td>Calibrator</td> <td>*0.735</td> <td></td> </tr> <tr> <td></td> <td>Calibrator</td> <td>(*Cal Value</td> <td>2.1</td> </tr> </table>				CALIBRATION	Spline	Water	0	<i>Vol aspirated</i> <i>Vol net</i> <i>Vol dil</i> 2.1   13   115 2.1   35   105 4.2   35   105 6.3   35   105 2.1   --   --		Calibrator	*0.1			Calibrator	*0.25			Calibrator	*0.489			Calibrator	*0.735			Calibrator	(*Cal Value	2.1
CALIBRATION	Spline	Water	0	<i>Vol aspirated</i> <i>Vol net</i> <i>Vol dil</i> 2.1   13   115 2.1   35   105 4.2   35   105 6.3   35   105 2.1   --   --																								
	Calibrator	*0.1																										
	Calibrator	*0.25																										
	Calibrator	*0.489																										
	Calibrator	*0.735																										
	Calibrator	(*Cal Value	2.1																									

The Calibration is stable until 27 days. After this period the Calibration must be performed again in order to obtain good results.

### PERFORMANCE CHARACTERISTICS

- Measurement range:** Up to 250 mg/dL, under the described assay conditions. Samples with higher concentrations, should be diluted 1/5 in NaCl 9 g/L and retested again. The linearity limit depends on the sample / reagent ratio. It will be higher by decreasing the sample volume, although the sensitivity of the test will be proportionally decreased.
- Detection Limit:** Values less than 1,91 mg/dL give non-reproducible results.
- Precision:** The reagent has been tested for 20 days, using three levels of serum in a EP5-based study (NCCLS).

EP5	CV (%)		
	23,92 mg/dL	59,08 mg/dL	119,07 mg/dL
Total	7,4%	4,3%	3,6%
Within Run	2,0%	1,4%	1,0%
Between Run	3,7%	2,2%	1,8%
Between Day	6,1%	3,4%	3,0%

- Accuracy:** Results obtained using this reagent (y) were compared to those obtained with a Daiichi immunoturbidimetric method. 48 samples ranging from 50 to 200 mg/dL of Apo B were assayed. The correlation coefficient (r) was 0,965 and the regression equation  $y = 0,996x + 5,112$ . The results of the performance characteristics depend on the used analyzer.

### BIBLIOGRAPHY

- Clinical Guide to Laboratory Tests, Edited by NW Tietz W B Saunders Co., Philadelphia, 483, 1983.
- Mahley RW et al. J Lipids Res 1984; 25: 1277-1294.
- Brown MS et al. Science 1986; 232:34-47.
- Freedman DS et al. N Eng J Med 1986; 315: 721-726.
- Sakurabayashi I et al. Clinica Chimica Acta 2001; 312: 87-95.
- Young DS. Effects of disease on clinical laboratory tests, 3th ed. AACC Pres, 1997.
- Friedman and Young. Effects of disease on clinical laboratory tests, 3th ed. AACC Pres, 1997.

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TEST INFORMATION		REAGENT VOLUME		
N°	**	Vol. R1	240	
Test	ApoB	Vol. R2	60	
Full Name	ApolipoproteinB	Vol. R3		
Standard n°	6	Vol. R4		
SAMPLE VOLUME		RESULT SETUP		
Vol. Sample Stand.	2	Decimal	1 Slope 1	
Vol. Sample Increas.		Unit	mg/dL Inter. 0	
Vol. Sample Dec				
REACTION PARAMETERS				
Reac. Type	End Point	Direction	Increase	
Pri. Wave.	340	Reagent Blank	40-41	
Sec. Wave.		React. Time	75-76	
CALIBRATION DILUTION (5 CAL + 1 WATER BLANK)				
CALIBRATION TYPE: SPLINE				
N° CAL DIL	CONCENTRATION	SAMPLE DIL	VOL DIL	VOL SAMPLE
0	CAL *0.1	10.0	90.0	2.0
1	CAL *0.25	30.0	90.0	2.0
2	CAL *0.4971	30.0	90.0	4.0
3	CAL *0.7412	30.0	90.0	6.0
4	CAL	0.0	0.0	2.0

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**BIBLIOGRAPHIE**

1. Clinical Guide to Laboratory Tests, Edited by NWTietz W B Saunders Co., Philadelphia, 483, 1983.
2. Mahley RW et al. J Lipids Res 1984; 25: 1277-1294.
3. Brown MS et al. Science 1986; 232:34-47.
4. Freedman DS et al. N Eng J Med 1986; 315: 721-726.
5. Sakurabayashi I et al. Clinica Chimica Acta 2001; 312: 87-95.
6. Young DS. Effects of disease on clinical laboratory tests, 3th ed. AACC Pres, 1997.
7. Friedman and Young. Effects of disease on clinical laboratory tests, 3th ed. AACC Pres, 1997.

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