Quantitative determination of immunoglobulin E (IgE) IVD

Store 2 - 8°C.

PRINCIPLE OF THE METHOD

The IgE-Turbilatex is a quantitative turbidimetric test for the measurement of IgE in human serum or plasma.

Latex particles coated with mouse IgG anti-human IgE are agglutinated when mixed with samples containing IgE. The agglutination causes an absorbance change, dependent upon the IgE contents of the patient sample that can be quantified by comparison from a calibrator of known IgE concentration.

CLINICAL SIGNIFICANCE

IgE is an immunoglobulin with a molecular weight of approximately 190,000 normally present in trace amounts. Continual production of IgE antibodies in response to common naturally occurring allergens, however, often results in elevated serum levels and in the development of such clinically important Type I allergic reactions as asthma, hay fever, dermatitis and food allergies. Elevated IgE levels are also seen in parasitic diseases, IgE myeloma, and in hepatitis. The measurement of IgE in human serum is thus considered to be useful in the diagnosis, treatment assessment of disease progression, or postoperative prognosis for such conditions.

REAGENTS

Diluent (R1)	Glycine buffer, pH 8,3. Preservative.
Latex (R2)	Latex particles coated with mouse IgG anti-human IgE, pH 7,3. Preservative.

CALIBRATION

The sensitivity of the assay and the target value of the calibrator have been standardized against the International Reference Preparation of IgE 75/502 (2nd IRP 1981) from WHO. It is not recommended the use of other commercially available IgE calibrators.

PREPARATION

Ready to use.

STORAGE AND STABILITY

All the components of the kit are stable until the expiration date on the label when stored tightly closed at 2-8°C and contaminations are prevented during their use. Reagents should not be left inside the analyzer after use, they must be stored refrigerated at 2-8°C. Latex may sediment. Mix reagents gently before use. Do not use reagents over the expiration date.

Frozen Latex and Diluent could change the functionality of the test.

Reagent deterioration: Presence of particles (R1, R2) and turbidity (R1).

ADDITIONAL EQUIPMENT

- SPIN640 / SPIN640Plus autoanalyzer
- Laboratory equipment.

SAMPLES

Fresh serum. Stable 7 days at 2-8°C or 3 months at -20°C.

The samples with presence of fibrin should be centrifuged before testing.

Do not use highly hemolized or lipemic samples.

PERFORMANCE CHARACTERISTICS

1. **Measurement range:** Up to 1000 IU/mL, under the described assay conditions. Samples with higher concentrations should be diluted 1/5 in NaCl 9 g/L and retested again. The linearity limit and measurement range depend on the sample to reagent / ratio, as well as the analyzer used. It will be higher by decreasing the sample volume, although the sensitivity of the test will be proportionally decreased.

2. **Limit detection:** Values less than 3,04 IU/mL give non-reproducible results.

3. **Prozone effect:** No prozone effect was detected upon 60.000 IU/mL.

4. **Sensitivity:** Δ 0,5 mA. IU/mL.

5. **Precision:**

		Within run		
		Mean (IU/mL)	SD (IU/mL)	CV (%)
n= 21		64,0	1,07	1,68
		121,0	1,43	1,18
		295,4	1,20	0,41
		491,3	4,01	0,82
		739,8	2,63	0,36

		Between days		
		Mean (IU/mL)	SD (IU/mL)	CV (%)
n= 21		61,0	2,9	4,76
		406	7,8	1,92

6. **Accuracy:** Results obtained using this reagents (y) were compared to those obtained using a nephelometric method (x). 92 samples were assayed by both methods. The correlation coefficient (r) was 0,9748 and the regression line equation $y = 1,0321x + 8,7369$.

The results of the performance characteristics depend on the analyzer used.

SPIN640 APPLICATION

TEST INFORMATION		REAGENT VOLUME		
Nº	**	Vol. R1	185	
Test	Ig E	Vol. R2	100	
Full Name	Immunoglobulin E	Vol. R3		
Standard nº	6	Vol. R4		
SAMPLE VOLUME		RESULT SETUP		
Vol. Sample Stand.	8	Decimal	1 Slope 1	
Vol. Sample Increas.		Unit	IU/mL Inter. 0	
Vol. Sample Dec				
REACTION PARAMETERS				
Reac. Type	Fixed Time	Direction	Increase	
Pri. Wave.	570	Reagent Blank	0-0	
Sec. Wave.		React. Time	44-80	
CALIBRATION DILUTION (4 CAL + 1 WATER BLANK)				
CALIBRATION TYPE: SPLINE				
Nº CAL DIL	CONCENTRATION	SAMPLE DIL	VOL DIL	VOL SAMPLE
0	CAL *0.125	15.0	90.0	8.0
1	CAL *0.25	30.0	90.0	8.0
2	CAL *0.4919	30.0	90.0	16.0
3	CAL	0.0	0.0	8.0

SPIN640Plus APPLICATION

EDIT PARAMETERS			
Test	IgE	No.	**
Full name	IgE	Print name	IgE
Reac. Type	Fixed Time	Direction	IncreasePri. Wave.
	570	Sec. Wave.	
Unit	UI/mL	Decimal	0.1
Reagent Blank	0 - 0	React. Time	51 - 82
Vol. Sample	8 ul	R1	185 ul
Increased		R2	100 ul
Decreased			R3
Sample blank		R4	
CALIBRATION			
Spline	Water	0	
Calibrator	*0.125	8	17.5 105
Calibrator	*0.25	8	35 105
Calibrator	*0.492	16	35 105
Calibrator	Cal Value	8	-- --

REFERENCE VALUES

Up to 350 IU/mL. Each laboratory should establish its own reference range.

QUALITY CONTROL

Control Sera are recommended to monitor the performance of manual and automated assay procedures. Spinreact Control Serum IgE is available (Ref.: 1107053).

Each laboratory should establish its own Quality Control scheme and corrective actions if controls do not meet the acceptable tolerances.

NOTES

Clinical diagnosis should be not made on findings of a single test result, but should integrate both clinical and laboratory data.

BARCODED REAGENTS LOAD MUST BE PRECEDED OF A SPINREACT "DATABASE" COPY INTO THE ANALYZER SOFTWARE. IT IS AVAILABLE UNDER REQUEST TO SPINREACT.

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MDTLIS07-F 25/01/21

APPLICATION AU SPIN640

TEST INFORMATION		REAGENT VOLUME		
N°	**	Vol. R1	185	
Test	Ig E	Vol. R2	100	
Full Name	Immunoglobulin E	Vol. R3		
Standard n°	6	Vol. R4		
SAMPLE VOLUME		RESULT SETUP		
Vol. Sample Stand.	8	Decimal	1 Slope 1	
Vol. Sample Increas.		Unit	IU/mL Inter. 0	
Vol. Sample Dec				
REACTION PARAMETERS				
Reac. Type	Fixed Time	Direction	Increase	
Pri. Wave.	570	Reagent Blank	0-0	
Sec. Wave.		React. Time	44-80	
CALIBRATION DILUTION (4 CAL + 1 WATER BLANK)				
CALIBRATION TYPE: SPLINE				
N° CAL DIL	CONCENTRATION	SAMPLE DIL	VOL DIL	VOL SAMPLE
0	CAL *0.125	15.0	90.0	8.0
1	CAL *0.25	30.0	90.0	8.0
2	CAL *0.4919	30.0	90.0	16.0
3	CAL	0.0	0.0	8.0

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