**Quantitative determination of Ferritin IVD**

Store 2 - 8°C.

**PRINCIPLE OF THE METHOD**

Ferritin-turbilatex is a quantitative turbidimetric test for the measurement of ferritin in human serum or plasma.

Latex particles coated with specific anti-human ferritin are agglutinated when mixed with samples containing ferritin. The agglutination causes an absorbance change, dependent upon the ferritin contents of the sample that can be quantified by comparison from a calibrator of known ferritin concentration.

**CLINICAL SIGNIFICANCE**

Serum ferritin concentration usually reflects body iron stores and is considered one of the most reliable indicators of iron status of patients. Whereas low serum concentrations of ferritin are always indicative of an iron deficiency, elevated concentrations can occur for variety of reasons. Thus, although elevated concentrations often indicate an excessive iron intake, they are also caused by liver disease, chronic inflammation and malignancies. Pregnant women, blood donors, hemodialysis patients, adolescents and children are groups particularly at risk.

**REAGENTS**

<b>Diluent (R1)</b>	Tris buffer 20 mmol/L, pH 8,2. Preservative.
<b>Latex (R2)</b>	Latex particles coated with rabbit IgG anti-human ferritin, pH 8,2. Preservative.
<b>FERR-CAL</b>	Calibrator. Ferritin concentration is stated on the vial label.

**PRECAUTIONS**

Components from human origin have been tested and found to be negative for the presence of HBsAg, HCV, and antibody to HIV (1/2). However, handle cautiously as potentially infectious.

**CALIBRATION**

Use Ferritin Calibrator Reference 1107042.

The sensitivity of the assay and the target value of the calibrator have been standardized against the 3rd International Standard of Ferritin (94/572, 2008 WHO). Recalibrate when control results are out of specified tolerances, when using different lot of reagent and when the instrument is adjusted.

**PREPARATION**
**Ferritin Calibrator:** Reconstitute (→) with 3,0 mL of distilled water. Mix gently and incubate at room temperature for about 10 minutes before testing.

**STORAGE AND STABILITY**

All the components of the kit are stable until the expiration date on the label when stored tightly closed at 2-8°C and contaminations are prevented during their use. Reagents should not be left inside the analyzer after use, they must be stored refrigerated at 2-8°C. Latex may sediment. Mix reagents gently before use. Do not use reagents over the expiration date. Do not freeze; frozen Latex or Diluent could change the functionality of the test.

**Reagent deterioration:** Presence of particles (R1, R2) and turbidity (R1).

**ADDITIONAL EQUIPMENT**

- SPIN640 / SPIN640Plus autoanalyzer
- Laboratory equipment.

**SAMPLES**

Fresh serum. Stable 7 days at 2-8°C or 3 months at -20°C.

The samples with presence of fibrin should be centrifuged before testing. Do not use highly hemolyzed or lipemic samples.

**REFERENCE VALUES**

Men: 30 – 220 µg/L.

Women: 20 – 110 µg/L.

Each laboratory should establish its own reference range.

**QUALITY CONTROL**

Control Sera are recommended to monitor the performance of manual and automated assay procedures. It should be used the SPINREACT Ferritin Control (Ref.: 1107044).

Each laboratory should establish its own Quality Control scheme and corrective actions if controls do not meet the acceptable tolerances.

**NOTES**

Clinical diagnosis should not be made on findings of a single test result, but should integrate both clinical and laboratory data.

**BARCODED REAGENTS LOAD MUST BE PRECEDED OF A SPINREACT "DATABASE" COPY INTO THE ANALYZER SOFTWARE. IT IS AVAILABLE UNDER REQUEST TO SPINREACT.**
**SPIN640 APPLICATION**

TEST INFORMATION		REAGENT VOLUME		
Nº	**	Vol. R1	160	
Test	FERR	Vol. R2	40	
Full Name	Ferritin	Vol. R3		
Standard nº	6	Vol. R4		
SAMPLE VOLUME		RESULT SETUP		
Vol. Sample Stand.	18	Decimal	1 Slope 1	
Vol. Sample Increas.		Unit	µg/L Inter. 0	
Vol. Sample Dec				
REACTION PARAMETERS				
Reac. Type	End Point	Direction	Increase	
Pri. Wave.	546	Reagent Blank	44-45	
Sec. Wave.		React. Time	73-74	
CALIBRATION DILUTION (5 CAL + 1 WATER BLANK)				
CALIBRATION TYPE: SPLINE				
Nº CAL DIL	CONCENTRATION	SAMPLE DIL	VOL DIL	VOL SAMPLE
1	CAL *0,0625	10,0	150	18,0
2	CAL *0,125	17,2	120	18,0
3	CAL *0,25	40,0	120	18,0
4	CAL *0,50	40,0	120	36,0
5	CAL *1,0			18,0

**SPIN640Plus APPLICATION**

EDIT PARAMETERS			
Test	FERR	No.	**
Full name	FERRITIN	Print name	FERR
Reac. Type	End Point	Direction	Increase
Pri. Wave.	546	Sec. Wave.	
Unit	µg/L	Decimal	0.1
Reagent Blank	51-52	React. Time	81 - 82
Vol. Sample	18 ul	R1	160 ul
Increased		R2	40 ul
Decreased		R3	
Sample blank		R4	
CALIBRATION			
Spline	Water	0	
Calibrator	*0.0625	18	10.0 150
Calibrator	*0.125	18	17.2 120
Calibrator	*0.25	18	40.0 120
Calibrator	*0.5	36	40.0 120
Calibrator	(* )Cal Value	18	-- --

 The Calibration is stable until **31 days**. After this period the Calibration must be performed again in order to obtain good results.

**PERFORMANCE CHARACTERISTICS**
**Measuring range:** Up to 600 µg/L. Samples with higher values should be diluted 1/5 in NaCl 9 g/L and retested. The upper linearity limit increases as the sample volume and the sensitivity decrease.

**Detection limit:** 5,04 µg/L.

**Quantification limit:** Values under 6,6 µg/L may give non-reproducible results.

**Prozone effect:** No prozone effect was detected at least up to 9000 µg/L.

**Precision:** According to the EP5-A2 standards (CLSI), the reagent has been tested for 20 days, measuring each level per duplicate twice a day (n=80):

Mean (µg/L)	Intra-assay (n= 80)			Total (n= 80)		
	33,4	114,5	289,8	33,4	114,5	289,8
SD	1,7	1,4	2,4	2,1	3,4	7,5
CV (%)	5,1	1,2	0,8	6,3	2,9	2,6

**Method comparison:** The reagent was compared to another commercially available Ferritin reagent by testing 144 samples (male and female), with concentrations between 6,97 and 730 µg/L. The coefficient of correlation (r) was 0,988, and the equation  $y = 0,96x + 1,15$ 

Performance characteristics depend on the analyzer used.

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 Vol. R1 160  
 Vol. R2 40  
 Vol. R3  
 Vol. R4

**SAMPLE VOLUME**

 Vol. Sample Stand. 18  
 Vol. Sample Inceas.  
 Vol. Sample Dec

**RESULT SETUP**

 Decimal 1 Slope 1  
 Unit µg/L Inter. 0

**REACTION PARAMETERS**

 Reac. Type End Point Direction Increase  
 Pri. Wave. 546 Reagent Blank 44-45  
 Sec. Wave. React. Time 73-74

**CALIBRATION DILUTION (5 CAL + 1 WATER BLANK)**

CALIBRATION TYPE: SPLINE

Nº CAL DIL	CONCENTRATION	SAMPLE DIL	VOL DIL	VOL SAMPLE
1	CAL *0,0625	10,0	150	18,0
2	CAL *0,125	17,2	120	18,0
3	CAL *0,25	40,0	120	18,0
4	CAL *0,50	40,0	120	36,0
5	CAL *1,0			18,0

**APLICACIÓN AL SPIN640Plus**
**EDIT PARAMETERS**

 Test FERR No. \*\*  
 Full name FERRITIN Print name FERR  
 Reac. Type End Point Direction Increase  
 Pri. Wave. 546 Sec. Wave.  
 Unit µg/L Decimal 0.1  
 Reagent Blank 51-52 React. Time 81 - 82  
 Vol. Sample 18 ul R1 160 ul  
 Increased R2 40 ul  
 Decreased R3  
 Sample blank R4

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### APPLICATION AU SPIN640

TEST INFORMATION		REAGENT VOLUME		
N°	**	Vol. R1	160	
Test	FERR	Vol. R2	40	
Full Name	Ferritin	Vol. R3		
Standard n°	6	Vol. R4		
SAMPLE VOLUME		RESULT SETUP		
Vol. Sample Stand.	18	Decimal	1 Slope 1	
Vol. Sample Increas.		Unit	µg/L Inter. 0	
Vol. Sample Dec				
REACTION PARAMETERS				
Reac. Type	End Point	Direction	Increase	
Pri. Wave.	546	Reagent Blank	44-45	
Sec. Wave.		React. Time	73-74	
CALIBRATION DILUTION (5 CAL + 1 WATER BLANK)				
CALIBRATION TYPE: SPLINE				
N° CAL DIL	CONCENTRATION	SAMPLE DIL	VOL DIL	VOL SAMPLE
1	CAL *0,0625	10,0	150	18,0
2	CAL *0,125	17,2	120	18,0
3	CAL *0,25	40,0	120	18,0
4	CAL *0,50	40,0	120	36,0
5	CAL *1,0			18,0

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