Quantitative determination of Immunoglobulin G (IgG) IVD

Store at 2 - 8°C.

PRINCIPLE OF THE METHOD

Anti-human IgG antibodies when mixed with samples containing IgG, form insoluble complexes. These complexes cause an absorbance change, dependent upon the IgG concentration of the patient sample, that can be quantified by comparison from a calibrator of known IgG concentration.

CLINICAL SIGNIFICANCE

IgG is the most important immunoglobulin produced by plasma cells, and represents about 75% of the total immunoglobulins. Its main function is to neutralize toxins in tissue spaces.

IgG deficit may be due to a congenital primary disturbance (immunodeficiency congenital and acquired) and is a special risk in children. Polyclonal hyperimmunoglobulinemia is the normal response to infections, especially in hepatitis and cirrhosis as well as autoimmune diseases. Increases of monoclonal IgG are found in multiple myeloma, lymphocytic leukemia, and Waldenström macroglobulinemia.

REAGENTS

R 1 Diluent	Tris buffer 20 mmol/L, PEG 8000, pH 8,3. Preservative.
R 2 Antibody	Goat serum, anti-human IgG, pH 7,5. Preservative.
Optional	Ref: 1102003 PROT CAL.

CALIBRATION

The assay is calibrated to the Reference Material ERM-DA470k/IFCC. It must be used the PROT CAL Calibrator to calibrate the reagent.

PREPARATION

Reagents: Ready to use.

STORAGE AND STABILITY

All the components of the kit are stable until the expiration date on the label when stored tightly closed at 2-8°C and contaminations are prevented during their use. Do not use reagents over the expiration date.

Reagent deterioration: The presence of particles and turbidity.

Do not freeze; frozen Antibody or Diluent could change the functionality of the test.

ADDITIONAL EQUIPMENT

- SPIN 800 autoanalyzer.
- Laboratory equipment.

SAMPLES

Fresh serum or plasma. EDTA or heparin should be used as anticoagulant. Stable 7 days at 2-8°C or 3 months at -20°C.

The samples with presence of fibrin should be centrifuged.

Do not use highly hemolyzed or lipemic samples.

REFERENCE VALUES

Between 700 - 1600 mg/dL. Each laboratory should establish its own reference range.

QUALITY CONTROL

Control sera are recommended to monitor the performance of manual and automated assay procedures. Spinreact PROT CONTROL (Cod.:1102004). Each laboratory should establish its own Quality Control scheme and corrective actions if controls do not meet the acceptable tolerances.

BARCODED REAGENTS LOAD MUST BE PRECEDED OF A SPINREACT "DATABASE" COPY INTO THE ANALYZER SOFTWARE. IT IS AVAILABLE UNDER REQUEST TO SPINREACT.

SPIN 800 APPLICATION

EDIT PARAMETERS			
Test	IgG	No.	**
Full name	Immunoglobulin G	Print name	IgG
Reac. Type	End Point	Direction	Increase
Pri. Wave.	605	Sec. Wave.	
Unit	mg/dL	Decimal	0,1
Reagent Blank	15 - 16	React. Time	26 - 27
Vol. Sample	2 ul	Increas.	Decrease
			Sample blank
R1	240 ul	R2	60 ul
		R3	R4
CALIBRATION			
Rule	Spline	Water	0
		Calibrator *0,1	2
		Calibrator *0,25	2
		Calibrator *0,497	4
		Calibrator *0,74	6
		Calibrator (*)Cal Value	2
			--
			--

Vol aspirated	Vol net	Vol dil
2	10	90
2	30	90
4	30	90
6	30	90
2	--	--

CAL AUTO DILUTIONS

The Calibration is stable until **31 days**. After this period the Calibration must be performed again in order to obtain good results.

PERFORMANCE CHARACTERISTICS

- Measurement range:** Up to 3000 mg/dL under the described assay conditions. Samples with higher concentrations, should be diluted 1/5 in NaCl 9 g/L and re-tested again. The linearity limit and measurement range depends on the sample to reagent / ratio. It will be higher by decreasing the sample volume, although the sensitivity of the test will be proportionally decreased.
- Detection Limit:** Values less than 10,3 mg/dL give non-reproducible results.
- Prozone effect:** No prozone effect was detected upon 8000 mg/dL
- Sensitivity:** Δ 0,6 mA. mg/dL at 359 mg/dL.
- Precision:** The reagent has been tested for 20 days, using three levels of serum in a EP5-based study.

EP5	CV (%)		
	340,3 mg/dL	801,96 mg/dL	1517,5 mg/dL
Total	2,1%	2,8%	4,8%
Within Run	0,9%	0,7%	1%
Between Run	1,5%	1,5%	1,8%
Between Day	1%	2,2%	4,4%

- Accuracy:** Results obtained using this reagent (y) were compared to those obtained using the Elecsys method from Roche. 79 samples ranging from 450 to 2600 mg/dL of IgG were assayed. The correlation coefficient (r) was 0.94 and the regression equation $y = 0,957x + 105,67$.

The results of the performance characteristics depend on the used analyzer.

NOTES

Clinical diagnosis should not be made on findings of a single test result, but should integrate both clinical and laboratory data.

BIBLIOGRAPHY

- Clinical Guide to Laboratory Tests, Edited by NW Tietz W B Saunders Co., Philadelphia, 483, 1983.
- Skoug Jonh W et al. Clin Chem 1988; 34/2: 309 - 315
- Pesce AJ and Kaplan, LA. Methods in Clinical Chemistry. The CV Mosby Company, St. Louis MO, 1987.
- Dati F et al. Eur J Clin Chem Clin Biochem 1966; 14: 401-406.
- Young DS. Effects of disease on clinical laboratory tests, 3th ed. AACC Pres, 1997
- Friedman and Young. Effects of disease on clinical laboratory tests, 3th ed. AACC Pres, 1997.

PACKAGING

Ref: MX1103004	Cont.	R1.Diluent:	2 x 40 mL
		R2.Antibody:	1 x 20 mL

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BIBLIOGRAFIA

- Clinical Guide to Laboratory Tests, Edited by NW Tietz W B Saunders Co., Philadelphia, 483, 1983.
- Skoug Jonh W et al. Clin Chem 1988; 34/2: 309 - 315
- Pesce AJ and Kaplan, LA. Methods in Clinical Chemistry. The CV Mosby Company, St. Louis MO, 1987.
- Dati F et al. Eur J Clin Chem Clin Biochem 1996; 34: 517-520.
- Young DS. Effects of disease on clinical laboratory tests, 3th ed. AACC Pres, 1997
- Friedman and Young. Effects of disease on clinical laboratory tests, 3th ed. AACC Pres, 1997.

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BIBLIOGRAPHIE

- Clinical Guide to Laboratory Tests, Edited by NW Tietz W B Saunders Co., Philadelphia, 483, 1983.
- Skoug Jonh W et al. Clin Chem 1988; 34/2: 309 - 315
- Pesce AJ and Kaplan, LA. Methods in Clinical Chemistry. The CV Mosby Company, St. Louis MO, 1987.
- Dati F et al. Eur J Clin Chem Clin Biochem 1996; 34: 517-520.
- Young DS. Effects of disease on clinical laboratory tests, 3th ed. AACC Pres, 1997.
- Friedman and Young. Effects of disease on clinical laboratory tests, 3th ed. AACC Pres, 1997.

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BIBLIOGRAFIA

1. Clinical Guide to Laboratory Tests, Edited by NW Tietz W B Saunders Co., Philadelphia, 483, 1983.
2. Skoug Jonh W et al. Clin Chem 1988; 34/2: 309 - 315
3. Pesce AJ and Kaplan, LA. Methods in Clinical Chemistry. The CV Mosby Company, St. Louis MO, 1987.
4. Dati F et al. Eur J Clin Chem Clin Biochem 1996; 34: 517-520.
5. Young DS. Effects of disease on clinical laboratory tests, 3th ed. AACC Pres, 1997
6. Friedman and Young. Effects of disease on clinical laboratory tests, 3th ed. AACC Pres, 1997.

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Ref: MX1103004

Cont.

R1.Solvente: 2 x 40 mL

R2.Anticorpo: 1 x 20 mL