α₁-Acid Glycoprotein

Turbidimetry

Quantitative determination of α₁-Acid Glycoprotein (α₁-Ac GLY)

IVD

Store at 2-8°C

PRINCIPLE OF THE METHOD

Anti- α₁-Ac Gly antibodies when mixed with samples containing α₁-Ac Gly, form insoluble complexes. These complexes cause an absorbance change, dependent upon the α₁-Ac Gly concentration of the patient sample, that can be quantified by comparison from a calibrator of known α₁-Ac Gly concentration.

CLINICAL SIGNIFICANCE

α₁-Ac Glycoprotein (also known as orosomucoid) is a glycoprotein synthesized by hepatic parenchymal cells, but granulocytes and monocytes may also contribute significantly to plasma levels in sepsis. It has long been known to bind a large number of basic and lipophilic compounds (progesterone and related hormones).

It is an acute phase response protein that shows a 3 to 4-fold increase in most conditions associated with inflammation or tissue necrosis, and may be one of the most reliable indicators of clinical activity of ulcerative colitis. Levels also are increased by glucocorticoid effect.

Synthesis and plasma levels are decreased by estrogens.

REAGENTS

R 1 Diluent	Tris buffer 20 mmol/L, PEG 8000, pH, 8.3. Preservative.
R 2 Antibody	Goat serum, anti-human α ₁ -Ac Glycoprotein, pH 7.5. Preservative.
Optional	Ref: 1102003 PROT CAL.

CALIBRATION

The assay has been standardized against the Reference Material ERM-DA470k/IFCC. It must be used the PROT CAL to calibrate the reagent.

PREPARATION

Reagents: Ready to use.

Calibration Curve: Prepare the following PROT CAL dilutions in NaCl 9 g/L as diluent. Multiply the concentration of the α₁-Ac Glycoprotein calibrator by the corresponding factor stated in table below to obtain the α₁-Ac Glycoprotein concentration of each dilution.

Calibrator dilution	1	2	3	4	5	6
Calibrator (μL)	--	10	25	50	75	100
NaCl 9 g/L (μL)	100	90	75	50	25	-
Factor	0	0,1	0,25	0,5	0,75	1,0

STORAGE AND STABILITY

All the components of the kit are stable until the expiration date on the label when stored tightly closed at 2-8°C and contaminations are prevented during their use. Do not use reagents over the expiration date.

Reagent deterioration: The presence of particles and turbidity.

Do not freeze; frozen Antibody or Diluent could change the functionality of the test.

ADDITIONAL EQUIPMENT

- MINDRAY BS-120 / BS-200E autoanalyzer
- Laboratory equipment.

SAMPLES

Fresh serum or plasma. EDTA or heparin should be used as anticoagulant. Stable 7 days at 2-8°C or 3 months at -20°C.

REFERENCE VALUES

Between 50 - 120 mg/dL. Each laboratory should establish its own reference range.

QUALITY CONTROL

Control sera are recommended to monitor the performance of manual and automated assay procedures. Spinreact PROT CONTROL (Ref.:1102004) is available. Each laboratory should establish its own Quality Control scheme and corrective actions if controls do not meet the acceptable tolerances.

NOTES

Clinical diagnosis should not be made on findings of a single test result, but should integrate both clinical and laboratory data.

MINDRAY BS-120 / BS-200E APPLICATION

PARAMETERS

Test	GLY / GLY	R1	240 / 240
Nº	**	R2	60 / 60
Full Name	GLY / GLY	Sample volume	3 / 3
Standard Nº	6 / 6	R1 Blank	
Reac. Type	Fixed T / Fixed T	Mixed Rgt Blank	
Pri. Wavelength	340 / 340	Linearity Range	10mg/dL 250mg/dL
Sec. Wavelength		Linearity Limit	*
Direction	Increase / Increase	Substrate Limit	*
Reac. Time	1_14 / -1_17	Factor	
Incuba. Time		Prozone check	*
Units	mg/dL / mg/dL	q1	q2
Precision	Interger / Interger	q3	q4
		PC	Abs

CALIBRATION (Cal + Rgt Blk)

BS120 (MANUAL) / BS200E (AUTODILUTION)

Rule	Spline / Spline
Sensitivity	1 / 1
Replicates	2 / 2
Interval (days)	0 / 0

BS200E CALIBRATION (DILUTION) → 5 CAL LEVELS + 1 WATER LEVEL

Nº CAL DIL	CONCENTRATION	DIL SAMPLE	DIL VOL	SAMPLE VOL
1	CAL *0.1	13.0	115	3.0
2	CAL *0.25	38.0	115	3.0
3	CAL *0.5	38.0	115	6.0
4	CAL *0.75	38.0	115	9.0
5	CAL*1			3.0

Blank parameter must be performed in order to get good results in CALIB screen from main menu. The blank calibration is stable until **35 days**. After this period the blank parameter must be performed again in order to validate the calibration.

PERFORMANCE CHARACTERISTICS

- Measurement range:** Up to 250 mg/dL under the described assay conditions. Samples with higher concentrations, should be diluted 1/5 in NaCl 9 g/L and retested again. The linearity limit depends on the sample / reagent ratio. It will be higher by decreasing the sample volume, although the sensitivity of the test will be proportionally decreased.
- Detection Limit:** Values less than 12,9 mg/dL give non-reproducible results.
- Prozone effect:** No prozone effect was detected upon 1000 mg/dL.
- Sensitivity:** Δ 5.0 mA / mg/dL.
- Precision:** The reagent has been tested for 20 days, using two levels of serum in a EP5-based study.

EP5	CV (%)	
	56,4 mg/dL	112,07 mg/dL
Total	3,3%	3,1%
Within Run	1,1%	1,6%
Between Run	3%	2,1%
Between Day	0,7%	1,6%

- Accuracy:** Results obtained using this reagent (y) were compared to those obtained using the method Immage from Beckman. 51 samples ranging from 50 to 120 mg/dL of α₁-Ac Gly were assayed. The correlation coefficient (r) was 0.95 and the regression equation
 $y = 0,9304x + 6,5367$.

The results of the performance characteristics depend on the used analyzer.

BIBLIOGRAPHY

- Clinical Guide to Laboratory Tests, Edited by NW Tietz W B Saunders Co., Philadelphia, 483, 1983.
- Dati F et al. Eur J Clin Chem Clin Biochem 1996; 34:517-520.
- Pesce AJ and Kaplan, LA. Methods in Clinical Chemistry. The CV Mosby Company, St. Louis MO, 1987.
- Bienvenue J et al. Clin Chem Clin Biochem 1981; 27: 721-726.
- Young DS. Effects of drugs on clinical laboratory tests, 4th ed. AACC Pres, 1995.
- Friedman and Young. Effects of disease on clinical laboratory tests, 3th ed. AACC Pres, 1997.

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BIBLIOGRAFIA

- Clinical Guide to Laboratory Tests, Edited by NW Tietz W B Saunders Co., Philadelphia, 483, 1983.
- Dati F et al. Eur J Clin Chem Clin Biochem 1996; 34:517-520.
- Pesce AJ and Kaplan, LA. Methods in Clinical Chemistry. The CV Mosby Company, St. Louis MO, 1987.
- Bienvue J et al. Clin Chem Clin Biochem 1981; 27: 721-726.
- Young DS. Effects of drugs on clinical laboratory tests, 4th ed. AACC Pres, 1995.
- Friedman and Young. Effects of disease on clinical laboratory tests, 3th ed. AACC Pres, 1997.

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BIBLIOGRAPHIE

1. Clinical Guide to Laboratory Tests, Edited by NW Tietz W B Saunders Co., Philadelphia, 483, 1983.
2. Dati F et al. Eur J Clin Chem Clin Biochem 1996; 34:517-520.
3. Pesce AJ and Kaplan, LA. Methods in Clinical Chemistry. The CV Mosby Company, St. Louis MO, 1987.
4. Bienvenue J et al. Clin Chem Clin Biochem 1981; 27: 721-726.
5. Young DS. Effects of drugs on clinical laboratory tests, 4th ed. AACC Pres, 1995.
6. Friedman and Young. Effects of disease on clinical laboratory tests, 3th ed. AACC Pres, 1997.

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