APO-B

Apolipoprotein B

Turbidimetry

Quantitative determination of apolipoprotein B (APO B) IVD

Store 2 - 8°C.

PRINCIPLE OF THE METHOD

Turbidimetric test for the measurement of apolipoprotein B in human serum or plasma.

Anti- Apo B antibodies when mixed with samples containing Apo B, form insoluble complexes. These complexes cause an absorbance change, dependent upon the Apo B concentration of the patient sample, that can be quantified by comparison from a calibrator of known Apo B concentration.

CLINICAL SIGNIFICANCE¹

Apo B is the major structural apolipoprotein in VLDL (Very Low Density Lipids), LDL (Low Density Lipids) lipoproteins and chylomicron. The most important function is the transport of rich tryglicerides lipoproteins from liver and intestine to other tissues. Apo B exists in two forms: Apo B-100 and Apo B-48. Apo B-100, the most important component of LDL, is synthesized in the liver and excreted in plasma as part of VLDL. Apo B-48, the most important component of chylomicrons, is synthesized in the intestine.

Several studies demonstrated that in people with coronary heart disease (CHD), changes in the serum concentrations of Apo A-I and Apo B are similar to those for HDL and LDL, respectively and whereas, are somewhat better discriminators of people with CHD than the LDL and HDL cholesterol concentrations.

The hiperbetalipoproteinemia is characterized by increased LDL Apo B-100 concentrations with normal or moderately increased concentrations of LDL cholesterol. The ratio of LDL cholesterol to Apo B-100 is therefore reduced in these patients.

Defects in the Apo B structure or lipoproteins containing Apo B prevent the secretion of triglycerides rich intestinal and hepatic lipoproteins; this disorder occurs in abetalipoproteinemia or homozygous hypobetalipoproteinemia.

REAGENTS

Diluent (R1)	Tris buffer 20 mmol/L, PEG, pH 8.3. Preservative.
Antibody (R2)	Goat serum, anti-human Apo B, tris 50 mmol/L, pH 7.5. Preservative.
Optional	APO CAL ref: 93005

CALIBRATION

The assay and the value of the calibrator concentration have been standardized against the Certified Reference Material WHO/IFCC SP3-07 (CDC, USA). It is recommended the use of the APO CAL Calibrator for calibration.

PREPARATION

Reagents: Ready to use.

Calibration Curve: Prepare the following APO CAL Calibrator dilutions in NaCl 9 g/L as diluent. Multiply the concentration of the Apo B calibrator by the corresponding factor stated in table below to obtain the Apo B concentration of each dilution.

Calibrator dilution	1	2	3	4	5	6
Calibrator (µL)	--	10	25	50	75	100
NaCl 9 g/L (µL)	100	90	75	50	25	-
Factor	0	0.1	0.25	0.5	0.75	1.0

STORAGE AND STABILITY

All the components of the kit are stable until the expiration date on the label when stored tightly closed at 2-8°C and contaminations are prevented during their use. Do not use reagents over the expiration date.

Reagent deterioration: The presence of particles and turbidity.

Do not freeze; frozen Antibody or Diluent could change the functionality of the test.

ADDITIONAL EQUIPMENT

- Spintech 240 autoanalyzer
- Laboratory equipment.

REFERENCE VALUES⁵

Between 69 – 105 mg/dL.

Each laboratory should establish its own reference range.

SAMPLES

Fresh serum or plasma. EDTA or heparin should be used as anticoagulant. Stable 2 weeks at 2-8°C or 3 months at -20°C.

The samples with presence of fibrin should be centrifuged before testing. Do not use highly hemolyzed or lipemic samples.

QUALITY CONTROL

Control sera are recommended to monitor the performance of manual and automated assay procedures. Spinreact Apolipoprotein Control (Ref.:93006) is available. Each laboratory should establish its own Quality Control scheme and corrective actions if controls do not meet the acceptable tolerances.

SPINTECH 240 APPLICATION

Item Name APO B		CALIBRATION	
DATA INFORMATION		TYPE	Spline
Units	mg/dL		
Decimals	0		
ANALYSIS		STANDARD	
Type	END	#1 0.10 x Cal. Val.	#4 0.75 x Cal. Val.
		#2 0.25x Cal. Val.	#5 1.00 x Cal. Val.
		#3 0.50 x Cal. Val.	#6
W.Length 1	340	NORMAL RANGE (37°C)	
		LOW	HIGH
Method	Turbidimetry	SERUM	MALE
			FEMALE
SLOPE	INTER	URINE	
1.000 x +	0		
Item Name APO B		DATA PROCESS	
ASPIRATION		ABSORBANCE LIMIT	
KIND	Single <input type="checkbox"/> Double <input checked="" type="checkbox"/>	READ	LOW -3.000
		START	HIGH 3.000
		END	
	VOLUME**	MAIN	42 43
SAMPLE	2 µL	SUB	30 31
REAGENT 1	240 µL	ENDPOINT LIMIT 3	
REAGENT 2	60 µL	LINEAR CHECK (%)	
		FACTOR	
Third Mix	<input checked="" type="checkbox"/> OFF <input type="checkbox"/> ON	Blank Correction	1.000
R1 Blank	<input checked="" type="checkbox"/> Water <input type="checkbox"/> R1-B	PROZONE CHECK	
MONITOR		START END LIMIT (%)	
0 LEVEL POINT	1	FIRST	
SPAN	3.000	SECOND	<input checked="" type="checkbox"/> Low High
		THIRD	<input checked="" type="checkbox"/> Low High

** Modify reagents and sample volumes according to the range accepted but keeping always the mentioned ratio.

INTERFERENCES

Hemoglobin (20 g/L), bilirubin (40 mg/dL), lipemia (5 g/L), and rheumatoid factor (800 UI/mL) do not interfere. Other substances may interfere ^{6,7}.

PERFORMANCE CHARACTERISTICS

- Measurement range:** Up to 250 mg/dL, under the described assay conditions. Samples with higher concentrations, should be diluted 1/5 in NaCl 9 g/L and retested again. The linearity limit depends on the sample / reagent ratio. It will be higher by decreasing the sample volume, although the sensitivity of the test will be proportionally decreased.
- Detection Limit:** Values less than 1,91 mg/dL give non-reproducible results.
- Precision:** The reagent has been tested for 20 days, using three levels of serum in a EP5-based study (NCCLS).

EP5	CV (%)		
	23,92 mg/dL	59,08 mg/dL	119,07 mg/dL
Total	7,4%	4,3%	3,6%
Within Run	2,0%	1,4%	1,0%
Between Run	3,7%	2,2%	1,8%
Between Day	6,1%	3,4%	3,0%

- Accuracy:** Results obtained using this reagent (y) were compared to those obtained with a Daiichi immunoturbidimetric method. 48 samples ranging from 50 to 200 mg/dL of Apo B were assayed. The correlation coefficient (r) was 0,965 and the regression equation $y = 0,996x + 5,112$.

The results of the performance characteristics depend on the used analyzer.

NOTES

Clinical diagnosis should not be made on findings of a single test result, but should integrate both clinical and laboratory data.

BIBLIOGRAPHY

- Clinical Guide to Laboratory Tests, Edited by NW Tietz W B Saunders Co., Philadelphia, 483, 1983.
- Mahley RW et al. J Lipids Res 1984; 25: 1277-1294.
- Brown MS et al. Science 1986; 232:34-47.
- Freedman DS et al. N Eng J Med 1986; 315: 721-726.
- Sakurabayashi I et al. Clinica Chimica Acta 2001; 312: 87-95.
- Young DS. Effects of disease on clinical laboratory tests, 3th ed. AACC Pres, 1997.
- Freedman and Young. Effects of disease on clinical laboratory tests, 3th ed. AACC Pres, 1997.

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