**Quantitative determination of β₂-microglobulin (β₂-m)
 IVD**

Store 2 - 8°C.

PRINCIPLE OF THE METHOD

The β₂-m Turbilatex is a quantitative turbidimetric test for the measurement of β₂-microglobulin (β₂-m) in human serum, plasma or urine. Latex particles coated with anti-human β₂-m are agglutinated when mixed with samples containing β₂-m. The agglutination causes an absorbance change, dependent upon the β₂-m contents of the patient sample that can be quantified by comparison from a calibrator of known concentration.

CLINICAL SIGNIFICANCE

β₂-m is a protein located on the surface of human lymphocytes and other nucleated cells. Free β₂-m is filtered by the glomerulus and subsequently reabsorbed in the proximal tubular cells. Increased urinary excretion of β₂-m is a sensitive indicator of renal insufficiency. Also, the β₂-m level in serum is a useful marker of other diseases including carcinomas, lymphoid tumors, rheumatoid arthritis and AIDS.

REAGENTS

β₂-m Diluent (R1)	Tris buffer 20 mmol/L, pH 8.2. Preservative.
β₂-m Latex (R2)	Particles coated with goat IgG anti-human β ₂ -m, pH 8,0. Preservative.
β₂-m CAL	Calibrator. β ₂ -m concentration is stated on the vial label.
Optional:	Ref: 1107034 β ₂ -m Control.

PRECAUTIONS

Components from human origin have been tested and found to be negative for the presence of HBsAg, HCV, and antibody to HIV (1/2). However, handle cautiously as potentially infectious.

CALIBRATION

Use β₂- Microglobulin Calibrator Reference 1107032. The sensitivity of the assay and the target value of the calibrator have been standardized against the 1st International ERM-DA470k/IFCC. The calibration in the SPINLAB 180 is stable for 1 month. Recalibrate when control results are out of specified tolerances, when using different lot of reagent and when the instrument is adjusted.

PREPARATION

Ready to use.

β₂-m Calibrator:

Serum method: Reconstitute (→) with 1.0 mL of distilled water. Mix gently and bring to room temperature for about 10 minutes before use.
 Urine method: Dilute reconstituted calibrator 1/6 with NaCl 9 g/L (50 μL calibrator + 250 μL NaCl 9 g/L).

STORAGE AND STABILITY

All the components of the kit are stable until the expiration date on the label when stored tightly closed at 2-8°C and contaminations are prevented during use. Reagents should not be left inside the analyzer after use, they must be stored refrigerated at 2-8°C. Latex may sediment. Mix reagents gently before use. Do not use reagents over the expiration date.

Do not freeze; frozen Latex or Diluent could change the functionality of the test.

Reagent deterioration: Presence of particles (R1, R2) and turbidity (R1).

β₂-m Calibrator: Stable for 1 month at 2-8°C or 3 months at -20°C.

ADDITIONAL EQUIPMENT

- Spintech 240 autoanalyzer
- Laboratory equipment.

SAMPLES

Fresh serum. Stable 7 days at 2-8°C or 3 months at -20°C.

Fresh urine. Adjust samples to pH 7-8 by the addition of K₂HPO₄. Stable 2 days at 2-8°C or 2 months at -20°C.

The samples with particles or fibrin should be centrifuged before testing. Do not use hemolyzed or lipemic samples.

REFERENCE VALUES

Serum: from 1.0 to 3.0 mg/L.

Urine: from 0.1 to 0.3 mg/L.

Each laboratory should establish its own reference range

QUALITY CONTROL

Control Sera are recommended to monitor the performance of manual and automated assay procedures. It should be used the SPINREACT β₂-m Control (Ref: 1107034). Each laboratory should establish its own Quality Control scheme and corrective actions if controls do not meet the acceptable tolerances.

SPINTECH 240 APPLICATION

Item Name B2m			
DATA INFORMATION		CALIBRATION	
Units	mg/L	TYPE	Linear
Decimals	1		
ANALYSIS		STANDARD	
Type	END	#1 *	#4
		#2	#5
		#3	#6
W.Length 1	546	NORMAL RANGE	
		LOW	HIGH
Method	Turbilatex	SERUM	MALE
			FEMALE
CORR		URINE (mg/24h)	
SLOPE	INTER		
1.000 x +	0		
Item Name B2m			
ASPIRATION		DATA PROCESS	
KIND	Single <input checked="" type="checkbox"/> Double	READ	ABSORBANCE LIMIT
		START	LOW -3.000
		END	HIGH 3.000
SAMPLE	VOLUME**	MAIN	44 45
REAGENT 1	3 μL	SUB	33 34
REAGENT 2	240 μL		
REAGENT 2	60 μL		ENDPOINT LIMIT 3
			LINEAR CHECK (%)
Third Mix	<input checked="" type="checkbox"/> OF ON	FACTOR	
R1 Blank	Water <input checked="" type="checkbox"/> R1-B	Blank Correction	1.000
MONITOR		PROZONE CHECK	
0 LEVEL POINT	1	START	END
SPAN	3.000	LIMIT (%)	
		FIRST	
		SECOND	<input checked="" type="checkbox"/> Low High
		THIRD	<input checked="" type="checkbox"/> Low High

** Modify reagents and sample volumes according to the range accepted but keeping always the mentioned ratio.

Blank parameter must be performed in order to get good results in CALIB screen from main menu. This parameter calibration is stable for 30 days

PERFORMANCE CHARACTERISTICS

- Linearity limit:** Up to 18 mg/L (serum) and 3 mg/L (urine), under the described assay conditions. Samples higher results should be diluted 1/5 in NaCl 9 g/L and retested again. The linearity depends on the sample-reagent ratio, as well as the analyzer used. It will be higher by decreasing sample volume, although the sensitivity of the test will be proportionally decreased.
- Detection limit:** Values less than 0,09 mg/L (serum) and 0,01 mg/L (urine) give non-reproducible results.
- Prozone effect:** No prozone effect was detected up to 100 mg/L (serum) and 20 mg/L (urine).
- Sensitivity:** Δ 0,048 A. mg/L (serum) and Δ 0,228 A. mg/L (urine).
- Precision:** The reagent has been tested for 20 days, using three different β₂-m concentrations in a EP5-based study.

EP5	CV (%)		
	+/- 1 mg/L	+/- 3,2 mg/L	+/- 8,5 mg/L
Total	4,0%	3,4%	1,7%
Within Run	2,8%	2,0%	1,2%
Between Run	1,7%	1,5%	1,2%
Between Day	2,2%	2,4%	0,0%

6. Accuracy: Results obtained using this reagent (y) were compared to those obtained using a commercial reagent (x) with similar characteristics. 36 samples of different concentrations of β₂-m were assayed. The correlation coefficient (r) was 0,97 and the regression equation y = 1,709x - 2,627.

The results of the performance characteristics depend on the analyzer used.

INTERFERENCES

Serum method: bilirubin (20 mg/L), hemoglobin (10 g/L) and lipids (10 g/L), do not interfere. Rheumatoid factors (150 IU/mL), interfere.

Urine method: urea (urine) (50 g/L), uric ac. (20 g/L) and glucose (100 g/L), do not interfere.

Other substances may interfere⁷.

BIBLIOGRAPHY

- Bhalla, R.B. et al. Clinical Chemistry 1983; 29: 1560.
- Malaguamera M et al. Digestive Diseases and Sciences 1997; 42: 762-766.
- Chironna et al. Int J Clin Lab Rws 1994; 24: 90-93.
- Wibell L et al. Nephron 1973; 10: 320-331.
- Berggard B et al. Scand J Clin Lab Invest 1980; 40: 13-25.
- Davey P G et al. Clin Chem 1982; 28/6: 1330-1333.
- Young DS. Effects of drugs on clinical laboratory test, 4th ed. AACC Pres, 1995.

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APPLICATION AU SPINTECH 240

Item Name B2m			
DATA INFORMATION			
Units	mg/L		
Decimals	1		
ANALYSIS			
Type	END		
W.Length 1	546		
Method	Turbilatex		
CORR			
SLOPE	INTER		
1.000 x +	0		
		CALIBRATION	
		TYPE	Linear
		STANDARD	
#1	*	#4	
#2		#5	
#3		#6	
		NORMAL RANGE	
SERUM	MALE	LOW	HIGH
	FEMALE		
URINE	(mg/24h)		
		DATA PROCESS	
ASPIRATION		ABSORBANCE LIMIT	
KIND	Single <input type="checkbox"/> Double <input checked="" type="checkbox"/>	READ	LOW -3.000
		START	HIGH 3.000
		END	
SAMPLE	VOLUME**	MAIN	44 45
REAGENT 1	3 µL	SUB	33 34
REAGENT 2	240 µL		
REAGENT 2	60 µL	ENDPOINT LIMIT 3	
		LINEAR CHECK (%)	
		FACTOR	
Third Mix	<input checked="" type="checkbox"/> OF <input type="checkbox"/> ON	Blank Correction 1.000	
R1 Blank	Water <input checked="" type="checkbox"/> R1-B		
MONITOR		PROZONE CHECK	
0 LEVEL POINT	1	START	END
SPAN	3.000	LIMIT (%)	
		FIRST	
		SECOND	<input checked="" type="checkbox"/> Low High
		THIRD	<input checked="" type="checkbox"/> Low High

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BIBLIOGRAPHIE

- Bhalla, R.B. et al. Clinical Chemistry 1983; 29: 1560.
- Malaguarrera M et al. Digestive Diseases and Sciences 1997; 42: 762-766.
- Chironna et al. Int J Clin Lab Rws 1994; 24: 90-93.
- Wibell L et al. Nephron 1973; 10: 320-331.
- Berggard B et al. Scand J Clin Lab Invest 1980; 40: 13-25.
- Davey P G et al. Clin Chem 1982; 28/6: 1330-1333.
- Young DS. Effects of drugs on clinical laboratory test, 4th ed. AACCPres, 1995.

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Item Name B2m		<u>DATA INFORMATION</u>		<u>CALIBRATION</u>	
Units	mg/L	TYPE	Linear		
Decimals	1				
<u>ANALYSIS</u>		<u>STANDARD</u>			
Type	END	#1	*	#4	
		#2		#5	
		#3		#6	
W.Length	1 546	<u>NORMAL RANGE</u>			
		SERUM	MALE	LOW	HIGH
Method	Turbilatex		FEMALE		
<u>CORR</u>		<u>URINE (mg/24h)</u>			
SLOPE	INTER				
1.000 x +	0				
Item Name B2m		<u>DATA PROCESS</u>		<u>ABSORBANCE LIMIT</u>	
<u>ASPIRATION</u>		<u>READ</u>		LOW -3.000	
KIND	Single <input type="checkbox"/> Double <input checked="" type="checkbox"/>	START	END	HIGH 3.000	
		MAIN	44 45		
SAMPLE	VOLUME**	SUB	33 34		
REAGENT 1	3 µL				
REAGENT 2	240 µL				
REAGENT 2	60 µL				
		ENDPOINT LIMIT 3			
		LINEAR CHECK (%)			
<u>MONITOR</u>		<u>FACTOR</u>			
Third Mix	<input checked="" type="checkbox"/> OF <input type="checkbox"/> ON	Blank Correction		1.000	
R1 Blank	Water <input checked="" type="checkbox"/> R1-B				
		<u>PROZONE CHECK</u>			
O LEVEL POINT 1		START		END LIMIT (%)	
SPAN 3.000		FIRST			
		SECOND		<input checked="" type="checkbox"/> Low High	
		THIRD		<input checked="" type="checkbox"/> Low High	

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BIBLIOGRAFIA

- Bhalla, R.B. et al. Clinical Chemistry 1983; 29: 1560.
- Malaguarnera M et al. Digestive Diseases and Sciences 1997; 42: 762-766
- Chironna et al. Int J Clin Lab Rws 1994; 24: 90-93
- Wibell L et al. Nephron 1973; 10: 320-331
- Berggard B et al. Scand J Clin Lab Invest 1980; 40: 13-25.
- Davey P G et al. Clin Chem 1982; 28/6: 1330-1333
- Young DS. Effects of drugs on clinical laboratory test, 4th ed. AAC Press, 1995.

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Ref.: TK1107030

Cont.

R1 Diluent :2 x 24 mL

R2 Latex :1 x 12 mL

 β₂-m CAL :1 x 1 mL