FOB-CAL
---------

**FOB Calibrator / Calibrador de FOB**  
Turbidimetry / Turbidimetria

**Quantitative determination of FOB**

IVD  
Store at 2 – 8 °C.

**PRODUCT CHARACTERISTICS**

The calibrator is a liquid solution containing a hemoglobin concentration suitable to calibrate fecal occult blood determinations in human samples by turbidimetric method.

**REAGENTS**

FOB-CAL	Human hemoglobin. Buffered solution. Preservative. Hemoglobin concentration is stated on the vial label.
---------	--

**PRECAUTIONS**

Components from human origin have been tested and found negative for the presence of HBsAg, HCV and antibody to HIV (1/2). However handle cautiously as potentially infectious.

**CALIBRATION**

The concentration value of the Calibrator has been standardized against the WHO International Standard Haemiglobincyanide NIBSC 98/708.

**PREPARATION**

Ready for use.

**STORAGE AND STABILITY**

The calibrator is stable until the expiration date on the label when stored tightly closed at 2-8 °C and contaminations are prevented during their use. Do not use reagents over the expiration date.

**PROCEDURE**

To be used in Spinreact's FOB-turbidimetric assay.

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FOB-CAL
---------

**FOB Calibrator / Calibrador de FOB**  
Turbidimetry / Turbidimetria

**Quantitative determination of FOB**

IVD  
Store at 2 – 8 °C.

**PRODUCT CHARACTERISTICS**

**The calibrator is a liquid solution containing a hemoglobin concentration suitable to calibrate fecal occult blood determinations in human samples by turbidimetric method.**

**REAGENTS**

FOB-CAL	Human hemoglobin. Buffered solution. Preservative. Hemoglobin concentration is stated on the vial label.
---------	--

**PRECAUTIONS**

Components from human origin have been tested and found negative for the presence of HBsAg, HCV and antibody to HIV (1/2). However handle cautiously as potentially infectious.

**CALIBRATION**

The concentration value of the Calibrator has been standardized against the WHO International Standard Haemiglobincyanide NIBSC 98/708.

**PREPARATION**

Ready for use

**STORAGE AND STABILITY**

The calibrator is stable until the expiration date on the label when stored tightly closed at 2-8 °C and contaminations are prevented during their use. Do not use reagents over the expiration date.

**PROCEDURE**

To be used in Spinreact's FOB-turbidimetric assay.

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