# MET Rapid Test Dipstick

(Urine)

## Package Insert

REF DME-101/111 English

A rapid test for the qualitative detection of Methamphetamine in human urine. For professional *in vitro* diagnostic use only.

### 【INTENDED USE】

The MET Rapid Test Dipstick is a rapid chromatographic immunoassay for the detection of methamphetamine in human urine at the cut-off concentration of 1,000ng/mL. This test will detect other compounds, please refer to analytical specificity table in this package insert.

This assay provides only a qualitative, preliminary test result. A more specific alternate chemical method must be used in order to obtain a confirmed result. Gas chromatography mass spectrometry (GC/MS) is the preferred confirmatory method. Clinical consideration and professional judgment should be applied to any drug of abuse test result, particularly when preliminary positive results are used.

### 【SUMMARY】

Methamphetamine is an addictive stimulant drug that strongly activates certain systems in the brain. Methamphetamine is closely related chemically to amphetamine, but the central nervous system effects of methamphetamine are greater. Methamphetamine is made in illegal laboratories and has a high potential for abuse and dependence. The drug can be taken orally, injected, or inhaled. Acute higher doses lead to enhanced stimulation of the central nervous system and induce euphoria, alertness, reduced appetite, and a sense of increased energy and power. Cardiovascular responses to methamphetamine include increased blood pressure and cardiac arrhythmias. More acute responses produce anxiety, paranoia, hallucinations, psychotic behavior, and eventually, depression and exhaustion.

The effects of methamphetamine generally last 2-4 hours, and the drug has a half-life of 9-24 hours in the body. Methamphetamine is excreted in the urine primarily as amphetamine, and oxidized and deaminated derivatives. However, 10-20% of methamphetamine is excreted unchanged. Thus, the presence of the parent compound in the urine indicates methamphetamine use. Methamphetamine is generally detectable in the urine for 3-5 days, depending on urine pH level.

The MET Rapid Test Dipstick is a rapid urine screening test that can be performed without the use of an instrument. The test utilizes a monoclonal antibody to selectively detect elevated levels of methamphetamine in urine. The MET Rapid Test Dipstick yields a positive result when the methamphetamine in urine exceeds 1,000ng/mL.

### 【PRINCIPLE】

The MET Rapid Test Dipstick is an immunoassay based on the principle of competitive binding. Drugs which may be present in the urine specimen compete against the drug conjugate for binding sites on the antibody.

During testing, a urine specimen migrates upward by capillary action. Methamphetamine, if present in the urine specimen below 1,000ng/mL, will not saturate the binding sites of the antibody coated particles in the test. The antibody coated particles will then be captured by immobilized methamphetamine conjugate and a visible colored line will show up in the test line region. The colored line will not form in the test line region if the methamphetamine level is at or above 1,000ng/mL because it will saturate all the binding sites of anti-methamphetamine antibodies.

A drug-positive urine specimen will not generate a colored line in the test line region because of drug competition, while a drug-negative urine specimen or a specimen containing a drug concentration less than the cut-off will generate a line in the test line region. To serve as a procedural control, a colored line will always appear at the control line region indicating that proper volume of specimen has been added and membrane wicking has occurred.

### 【REAGENTS】

The test contains mouse monoclonal anti-methamphetamine antibody-coupled particles and methamphetamine-protein conjugate. A goat antibody is employed in the control line system.

### 【PRECAUTIONS】

- For medical and other professional *in vitro* diagnostic use only. Do not use after the expiration date.
- The test should remain in the sealed pouch until use.
- All specimens should be considered potentially hazardous and handled in the same manner as an infectious agent.
- The used test should be discarded according to local regulations.

### 【STORAGE AND STABILITY】

Store as packaged at room temperature or refrigerated (2-30°C). The test is stable through the expiration date printed on the sealed pouch or label of the closed canister. The test must remain in the sealed pouch or closed canister until use. **DO NOT FREEZE.** Do not use beyond the expiration date.

**NOTE:** Once the canister has been opened, the remaining test(s) are stable for 50 days only.

### 【SPECIMEN COLLECTION AND PREPARATION】

#### Urine Assay

The urine specimen must be collected in a clean and dry container. Urine collected at any time of the day may be used. Urine specimens exhibiting visible particles should be centrifuged, filtered, or allowed to settle to obtain clear specimen for testing.

#### Specimen Storage

Urine specimens may be stored at 2-8°C for up to 48 hours prior to testing. For long-term storage, specimens may be frozen and stored below -20°C. Frozen specimens should be thawed and mixed before testing.

### 【MATERIALS】


- Test dipsticks
  - Package insert
- Specimen collection containers
  - Timer

#### Materials Required But Not Provided

### 【DIRECTIONS FOR USE】

**Allow the test, urine specimen and/or controls to reach room temperature (15-30°C) prior to testing.**

- Bring the pouch to room temperature before opening it. Remove the test dipstick from the sealed pouch and use it within one hour.
- With arrows pointing toward the urine specimen, **immerse the test dipstick vertically in the urine specimen for at least 10-15 seconds.** Do not pass the maximum line (MAX) on the test dipstick when immersing the strip. See the illustration below.
- Place the test dipstick on a non-absorbent flat surface, start the timer and wait for the colored line(s) to appear. **Read results at 5 minutes.** Do not interpret the result after 10 minutes.

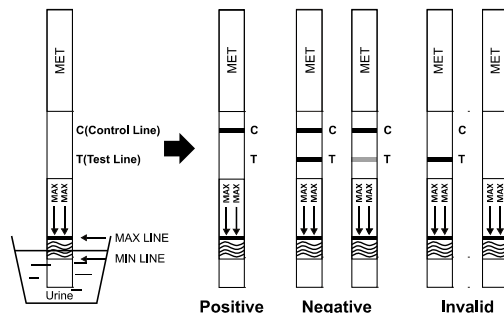### 【INTERPRETATION OF RESULTS】

(Please refer to the illustration above)

**NEGATIVE:** Two colored lines appear. One color line should be in the control region (C) and another color line should be in the test region (T). This negative result indicates that the methamphetamine concentration is below the detectable level of 1,000ng/mL.

**\*NOTE:** The shade of color in the test region (T) may vary, but it should be considered negative whenever there is even a faint color line.

**POSITIVE:** One color line appears in the control region (C). No line appears in the test region (T). This positive result indicates that the methamphetamine concentration is above the detectable level of 1,000ng/mL.

**INVALID:** Control line fails to appear. Insufficient specimen volume or incorrect procedural techniques are the most likely reasons for control line failure. Review the procedure and repeat the test with a new test. If the problem persists, discontinue using the test dipstick immediately and contact your local distributor.

### 【QUALITY CONTROL】

A procedural control is included in the test. A color line appearing in the control region (C) is considered an internal procedural control. It confirms sufficient specimen volume and correct procedural technique.

Control standards are not supplied with this kit; however it is recommended that positive and negative controls be tested as good laboratory testing practices to confirm the test procedure and to verify proper test performance.

### 【LIMITATIONS】

- The MET Rapid Test Dipstick provides only a qualitative, preliminary result. A secondary analytical method must be used to obtain a confirmed result. Gas chromatography mass spectrophotometry (GC/MS) is the preferred confirmatory method.<sup>1,2</sup>
- It is possible that technical or procedural errors, as well as other interfering substances in the urine specimen may cause erroneous results.
- Adulterants, such as bleach and/or alum, in urine specimens may produce erroneous results regardless of the analytical method used. If adulteration is suspected, the test

should be repeated with another urine specimen.

- A positive result indicates presence of the drug or its metabolites but does not indicate level of intoxication, administration route or concentration in urine.
- A negative result may not necessarily indicate drug-free urine. Negative results can be obtained when drug is present but below the cut-off level of the test.
- Test does not distinguish between drugs of abuse and certain medications.

### 【PERFORMANCE CHARACTERISTICS】

#### Accuracy

A side-by-side comparison was conducted using the MET Rapid Test Dipstick and a commercially available MET rapid test. Testing was performed on 107 clinical specimens previously collected from subjects present for drug screen testing. The following results were tabulated:

Method	Other MET Rapid Test		Total Results
	Positive	Negative	
MET Rapid Test Dipstick	Positive	55	55
	Negative	0	52
<b>Total Results</b>	55	52	107
<b>% Agreement</b>	>99.9%	>99.9%	>99.9%

A side-by-side comparison was conducted using the MET Rapid Test Dipstick and GC/MS at the cut-off of 1,000ng/mL. Testing was performed on 250 clinical specimens previously collected from subjects present for drug screen testing. The following results were tabulated:

Method	GC/MS		Total Results
	Positive	Negative	
MET Rapid Test Dipstick	Positive	76	81
	Negative	3	166
<b>Total Results</b>	79	171	250
<b>% Agreement</b>	96.2%	97.1%	96.8%

#### Analytical Sensitivity

A drug-free urine pool was spiked with methamphetamine at the following concentrations: 0 ng/mL, 500 ng/mL, 750 ng/mL, 1,000 ng/mL, 1,250 ng/mL, 1,500 ng/mL and 3,000 ng/mL. The result demonstrates >99% accuracy at 50% above and 50% below the cut-off concentration. The data are summarized below:

Methamphetamine Concentration (ng/mL)	Percent of Cut-off	n	Visual Result	
			Negative	Positive
0	0	30	30	0
500	-50%	30	30	0
750	-25%	30	26	4
1,000	Cut-off	30	14	16
1,250	+25%	30	3	27
1,500	+50%	30	0	30
3,000	3X	30	0	30

#### Analytical Specificity

The following table lists compounds that are positively detected in urine by the MET Rapid Test Dipstick at 5 minutes.

Compound	Concentration (ng/mL)
p-Hydroxymethamphetamine	25,000
D-Methamphetamine	1,000
L-Methamphetamine	20,000
(±)-3,4-Methylenedioxymethamphetamine	12,500
Mephentermine	50,000

#### Precision

A study was conducted at three physicians' offices using three different lots of product to demonstrate the within run, between run and between operator precision. An identical panel of coded specimens containing, according to GC/MS, no methamphetamine, 25% methamphetamine above and below the cut-off, and 50% methamphetamine above and below the 1,000 ng/mL cut-off was provided to each site.

Methamphetamine Concentration (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
500	10	10	0	10	0	10	0
750	10	9	1	9	1	9	1
1,250	10	1	9	2	8	1	9
1,500	10	0	10	0	10	0	10

#### Effect of Urinary Specific Gravity

Fifteen urine specimens of normal, high and low specific gravity ranges were spiked with 500 ng/mL and 1,500 ng/mL of methamphetamine. The MET Rapid Test Dipstick was tested in duplicate using the fifteen neat and spiked urine specimens. The results demonstrate that

varying ranges of urinary specific gravity do not affect the test results.

#### Effect of Urinary pH

The pH of an aliquoted negative urine pool was adjusted to a pH range of 5 to 9 in 1 pH unit increments and spiked with methamphetamine to 500 ng/mL and 1,500 ng/mL. The spiked, pH-adjusted urine was tested with the MET Rapid Test Dipstick in duplicate. The results demonstrate that varying ranges of pH do not interfere with the performance of the test.

#### Cross-Reactivity

A study was conducted to determine the cross-reactivity of the test with compounds in either drug-free urine or methamphetamine positive urine. The following compounds show no cross-reactivity when tested with the MET Rapid Test Dipstick at a concentration of 100 µg/mL.

#### Non Cross-Reacting Compounds

4-Acetamidophenol	Creatinine	Loperamide
Acetophenetidin	Deoxycorticosterone	Maprotiline
N-Acetylprocainamide	Dextromethorphan	Meperidine
Acetylsalicylic acid	Diazepam	Meprobamate
Aminopyrine	Diclofenac	Methadone
Amitriptyline	Diflunisal	Methoxyphenamine
Amobarbital	Digoxin	(+) 3,4-Methylenedioxy-
Amoxicillin	Diphenhydramine	amphetamine
Ampicillin	Doxylamine	3,4-Methylenedioxyethyl-
L-Ascorbic acid	Ecgonine hydrochloride	amphetamine
D-Amphetamine	Ecgonine methylester	Methylphenidate
D,L-Amphetamine	(1R,2S)-(-)-Ephedrine	Morphine-3-β-D-
L-Amphetamine	L-Epinephrine	glucuronide
Apomorphine	(-)-ψ-Ephedrine	Nalidixic acid
Aspartame	Erythromycin	Naloxone
Atropine	β-Estradiol	Naltrexone
Benzilic acid	Estrone-3-sulfate	Naproxen
Benzoic acid	Ethyl-p-aminobenzoate	Niacinamide
Benzoylcegonine	Fenfluramine	Nifedipine
Benzphetamine	Fenoprofen	Norethindrone
Bilirubin	Furosemide	D-Norpropoxyphene
(±)-Brompheniramine	Gentisic acid	Noscapine
Caffeine	Hemoglobin	D,L-Octopamine
Cannabidiol	Hydralazine	Oxalic acid
Chloralhydrate	Hydrochlorothiazide	Oxazepam
Chloramphenicol	Hydrocodone	Oxolinic acid
Chlordiazepoxide	Hydrocortisone	Oxycodone
Chlorothiazide	p-Hydroxyamphetamine	Oxymetazoline
(±) Chlorpheniramine	O-Hydroxyhippuric acid	Papaverine
Chlorpromazine	3-Hydroxytyramine	Penicillin-G
Chlorquine	Ibuprofen	Pentobarbital
Cholesterol	Imipramine	Perphenazine
Clomipramine	Iproniazid	Phencyclidine
Clonidine	(±)-Isoproterenol	Phenelzine
Cocaeethylene	Isoxsuprine	Phenobarbital
Cocaine hydrochloride	Ketamine	Phentermine
Codeine	Ketoprofen	L-Phenylephrine
Cortisone	Labetalol	L-Phenylethylamine
(-) Cotinine	Levorphanol	Phenylpropanolamine
Prednisolone	Zomepirac	Tolbutamine
Prednisone	Sulfamethazine	Trans-2- phenyl
Procaine	Sulindac	cyclopropylamine
Promazine	Temazepam	Triamterene
Promethazine	Tetracycline	Trifluoperazine
D,L-Propranolol	Tetrahydrocortisone,	Trimethoprim
D-Propoxyphene	3-Acetate	Trimipramine
D-Pseudoephedrine	Tetrahydrocortisone	Tryptamine
Quinacrine	3-(β-D glucuronide)	D, L-Tryptophan
Quinidine	Tetrahydrozoline	Tyramine
Quinine	Thiamine	Uric acid
Ranitidine	Thioridazine	Verapamil
Salicylic acid	D, L-Tyrosine	Serotonin
Secobarbital	(5-Hydroxytyramine)	

#### 【BIBLIOGRAPHY】

- Hawks RL, CN Chiang. Urine Testing for Drugs of Abuse. National Institute for Drug Abuse (NIDA), Research Monograph 73, 1986
- Baselt RC. Disposition of Toxic Drugs and Chemicals in Man. 2nd Ed. Biomedical Publ., Davis, CA. 1982; 488

#### Index of Symbols

	Consult instructions for use or consult electronic instructions for use		Contains sufficient for <n> tests		Temperature limit
	In vitro diagnostic medical device		Batch code		Catalogue number
	Authorized representative in the European Union		Use-by date		Do not re-use
	Do not use if package is damaged and consult instructions for use		Manufacturer		Importer

**Hangzhou AllTest Biotech Co.,Ltd.**  
 #550, Yinhai Street  
 Hangzhou Economic & Technological Development Area  
 Hangzhou, 310018 P.R. China  
 Web: www.alltests.com.cn Email: info@alltests.com.cn

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MedNet EC-REP GmbH
**SPINREACT, S.A.U.**