AMMONIA ETHANOL CONTROL

CONTROL AMONIACO ETANOL
Human Control serum
Suero Control humano

**NH3/EtOH Control serum
IVD**

Store at 2 - 8°C

PRODUCT CHARACTERISTICS

The Ammonia Ethanol control is a human liquid control serum. Is intended for quality control procedures in examining the accuracy and precision of quantitative Ammonia and Alcohol assays.

For *in vitro* diagnostic use only.

PRECAUTIONS

Each donor unit in the preparation of this material was tested by FDA-approved method and found negative for Human Immunodeficiency Virus (HIV 1, HIV 2) antibody, Hepatitis B Surface Antigen (HbsAg), and Hepatitis C Virus (HCV) antibody. However this material and all patient samples should be handled as if potentially infectious and disposed accordingly.

PREPARATION

Ready for use.

STORAGE AND STABILITY

The Control serum is stable until the expiration date on the label when stored tightly closed at 2-8°C and contaminations are prevented during their use.

Once opened, the product is stable for 30 days at 2-8°C. Only the required amount of product should be removed. After use, any residual product should NOT BE RETURNED to the original vial.

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*This Values sheet is applicable to lot and sublots. Sequential alphabetical letter (e.g. A, B, C etc.) following the lot number.

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